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Model:

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En
INSTRUCTIONS FOR USE
TOILET STOOL FOR SHOWER: AT01004, AT01005

INTENDED USE:

The shower stool is designed for people with limited mobility to facilitate the use of the shower.

The stool has adjustable legs and a non-slip seat to provide maximum comfort to the user. Before each use, check that the adjustment is made correctly.

After the user uses the stool, take care to clean the product thoroughly. Standard cleaning and disinfecting agents can be used.

AT01004 Shower toilet stool

- Aluminum toilet stool with plastic seat
- feet finished with anti-slip rubber caps with increased surface contact with the ground guarantee high stability
- seat height adjustable

Technical parameters:

- weight: 1.5 kg
- seat height: 37-54 cm
- width: 45 cm
- seat diameter: 31 cm Maximum load: 130 kg

AT01005 Toilet stool for shower with backrest

- aluminum toilet stool with plastic seat and backrest (detachable backrest)
- feet finished with anti-slip rubber caps with increased surface contact with the ground guarantee high stability
- seat height adjustable

- Polish product Technical parameters:

- weight: 1.5 kg
- seat height: 37-54 cm
- width: 45 cm
- seat diameter: 31 cm

Maximum load: 130 kg

CAUTION:

Take special care when using the product. Check that the product is properly assembled. Do not lean forward or backward excessively, as this may cause the product to tip over.

CAUTION:

In the event of a product-related “serious incident” that has directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

HOW TO DISPOSE OF THE DEVICE AFTER DECOMMISSIONING

After the device is withdrawn from service, the medical device can be disposed of as ordinary municipal waste.

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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Antar Medizin GmbH