AT02001, AT02002, AT02003

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UŻYTKOWANIA

WALKER USER MANUAL

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MANUAL DE USO DEL BALCÓN

MANUALE D'USO DEL BALCONE

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IU_AT02001, AT02002, AT02003

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Thank you for purchasing our product. Before performing any operations, check the condition of the device. If the product works or looks abnormal, contact the retailer where you purchased the device immediately. Do not use the product before reading and understanding this manual. If you do not understand any warnings, cautions, or instructions, contact your healthcare professional or retailer to avoid damaging the product.

NOTE: Inspect all parts for shipping damage. If you notice such damage, DO NOT use the product. More information from the manufacturer.

DESCRIPTION AND INTENDED USE

Walkers are light, easy-to-use products, designed to help people with limited mobility in moving around. They provide the user with support, reducing the risk of tipping over, and also enable walking with support on both sides.

MODELS:

Depending on the needs and preferences of the user, it is possible to choose one of the 3 versions of the walkers:

AT02001 – version with 4 swivel wheels. The product allows the user to move quickly in all directions, and if necessary, it is possible to block the walker by applying the brakes. This model is recommended for people with high mobility, who do not have problems with balance.

AT02002 – version with 2 wheels, walking. It allows you to move the walker without having to lift it.

AT02003 – version with 4 feet, adjustable, walking. Provides the greatest stability. In order to move the walker, it is necessary to lift it completely (when it is locked) or its side - when the walker is in the movable option. It is recommended mainly for moving on short distances.

CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product

WARNINGS:

- 1) Do not use the AT02001 for people with balance disorders. This may lead to the user falling over.
- 2) Do not lean out of the walker. This may cause the user to fall over.
- 3) Do not swing the walker. This may cause the user to fall over.
- 4) If the walker is four-wheeled (AT02001), if it is not used at the moment - its brakes should be turned on.
- 5) Do not use if user weight exceeds 150 kg.
- 6) Avoid contact with open fire. The product may ignite.
- 7) Make sure that the steel pin of the foot or caster clip goes through the hole on both sides of the wall after installation. Otherwise, the product may not be stable and the hole may become deformed. Also, the pin that changes the walker from a walker to a fixed one should pass through the hole on both sides of the wall.
- 8) Do not use the 4-wheel walker (AT02001) in the walking mode. It should always be a permanent option.

ATTENTION: it is forbidden to use the product in a way other than for its intended purpose

ATTENTION: Before use, check that all elements have been assembled correctly.

NOTE: The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

NOTE: If the product is used incorrectly, there may be a risk of tipping over. Please follow the boarding/dismounting/moving instructions.

TARGET PATIENT GROUP

People suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/ necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- (c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where


the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION:


In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

PREPARATION FOR USE:

The walker is delivered folded for transport.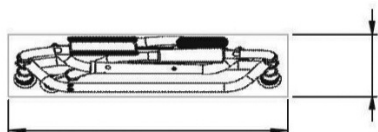

Folded walker

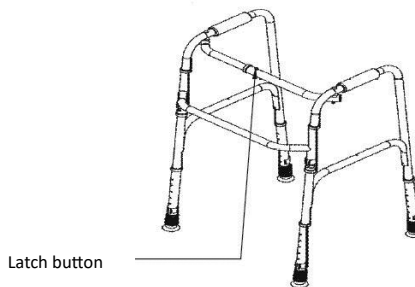Latch button

UNFOLDING

The walker is unfolded by grabbing its sides and swinging them outwards. After unfolding the walker, the latch button in the upper, central part of the walker should be in the hole and lock the walker in the unfolded position.

FOLDING

To fold the walker, press the latch button and pull the sides of the walker inwards, and then fold the walker.

HEIGHT ADJUSTMENT

Adjust the height of the product before using it. To do this, remove the clip, adjust height - the same for all four legs, and then insert the clip into the appropriate hole. **MAKE SURE THE STEEL PIN GOES THROUGH THE HOLE ON BOTH SIDES.** Otherwise, the product may not be stable and the hole may become deformed.

USE

The AT02002 and AT02003 can be used in a fixed or walking option. The walking option means that the sides of the walker slide alternately, allowing the walker to be moved forward by lifting only one side. In the fixed option, the balcony becomes rigid. It can be moved by lifting the entire walker and moving it forward.


In order to change the walker from the walking option to the fixed option, remove the clips on the sides of the walker and install them in the appropriate holes in the front part. Walker in models AT02002 and AT02003 in fixed option no can be submitted. To fold it, you need to switch it to the rolling option.

MOVING WITH THE BALCONY

AT02001 – The walker moves in any direction by pushing it. IF AT THE MOMENT THE USER DOESN'T USE THE PRODUCT, **ACTIVATE (BY PRESSING) THE BRAKES ON THE WHEELS.**

AT02002 – The walker moves forward by lifting its back, pushing it and putting the rear part on the ground.

AT02003 - The walker is moved by lifting it and placing it slightly in front, and in the walking option by lifting 1 side of the walker and pushing it forward, then repeat the steps with the other side.

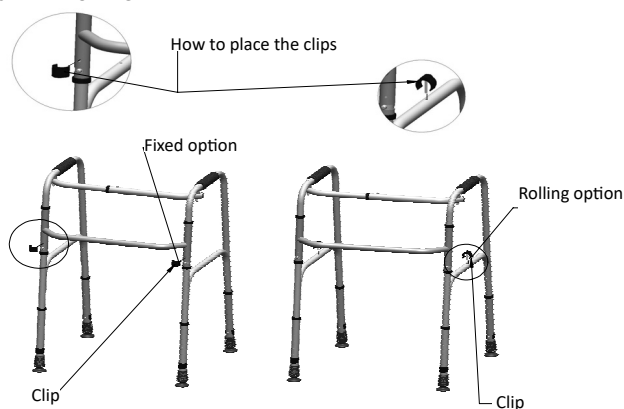CHANGE OPTIONS: FIXED-ROLLING**SPECIFICATIONS AT02001**

Height: 87-99 cm
 Width (at widest point - external dimension): 56 cm
 Width when folded: 15 cm
 Width between handles: 41 cm
 Weight: 4.2 kg
 Wheel diameter: 75 mm
 Maximum user weight: 150 kg

SPECIFICATION AT02002

Height: 80-92 cm
 Width (at widest point - external dimension): 56 cm
 Width when folded: 15 cm
 Width between handles: 41 cm
 Weight: 3 kg
 Wheel diameter: 75 mm
 Maximum user weight: 150 kg

SPECIFICATION AT02003

Height: 79- 93 cm
 Width (at widest point - external dimension): 56 cm
 Width when folded: 15 cm
 Width between handles: 41 cm
 Weight: 2.75 kg
 Maximum user weight: 150 kg

This symbol indicates the user's maximum weight

SCOPE OF DELIVERY: Walker, instruction manual

PRODUCT MAINTENANCE AND CLEANING

The product should be cleaned periodically with water and cleaning agents. The product can be treated with disinfectants. Rough objects and solvents must not be used for cleaning.

STORAGE

When not in use, the product should be stored in a dry and cool place away from direct sunlight. Avoid prolonged exposure to direct sunlight.

DISPOSAL OF THE PRODUCT

After withdrawal from use, the product can be treated as municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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