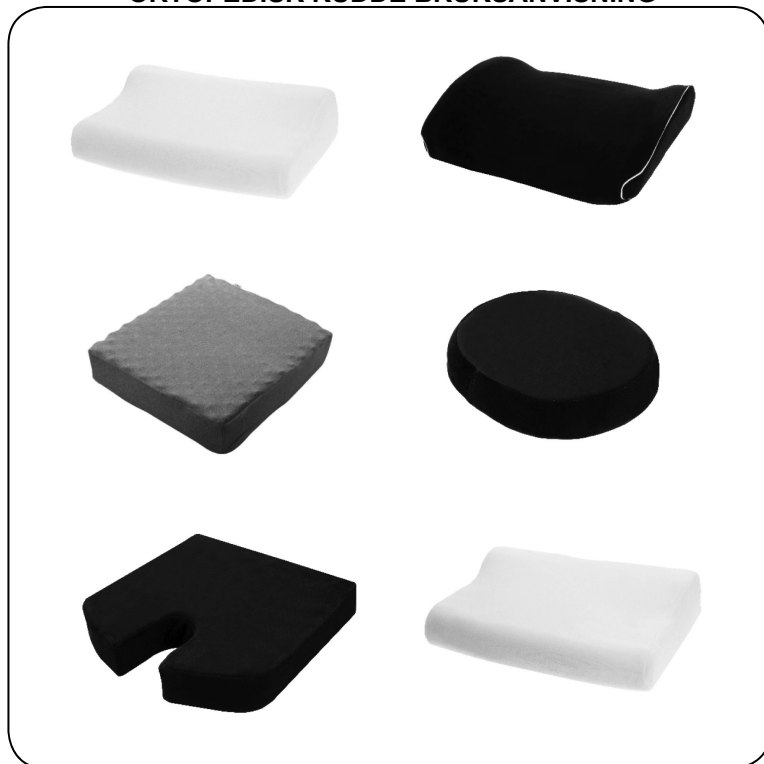
**AT03001, AT03002, AT03003, AT03004, AT03005,  
AT03006, AT03006 D, AT03007, AT03008, AT03009, AT03010,  
AT03011, AT03012, AT03013, AT03014, AT03015, AT03016,  
AT03017 PODUSZKA ORTOPEDYCZNA INSTRUKCJA**

**UŻYTKOWANIA**

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### GENERAL WARNING

Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact your healthcare professional or dealer to avoid damage to the product.

### PURPOSE/ INDICATIONS

Depending on the type of pillow, they have different uses. The sleeping pillow is primarily intended for people who suffer, for example, from tension and migraine headaches and pain and strain syndromes of the spine.

The task of the lumbar pillow is to maintain the natural position of the spine, supporting it while maintaining a prolonged sitting position. Seat cushions prevent pains caused by prolonged sitting, especially in wheelchair users.

### CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

### DIMENSIONS

AT03001: length 44 cm, width 30 cm, height 7 to 11 cm

AT03002: length 44 cm, width 30 cm, height 7 to 11 cm

AT03003: length 42 cm, width 35 cm, height 7 to 10 cm

AT03004: Width 33 cm, Height 22 cm, Thickness 7 cm

AT03005: width 33 cm, length 33 cm, height 11 cm

AT03006/AT03006 D (in disinfection cover):

- S: 42 x 38 x 8 cm,

- M: 43 x 43 x 8 cm,

- L: 46 x 43 x 8 cm,

- XL: 48 x 43 x 8 cm

AT03007: length: 47 cm, width: 32 cm, height: 7 - 12 cm

AT03008: 45 x 40 x 10 cm, 45 x 40 x 6 cm

AT03009: 41 x 32 x 8 cm

AT03010: 45 x 40 x 10 cm, 45 x 40 x 6 cm

AT03011:

- S: 42 x 38 x 8 cm,

- M: 43 x 43 x 8 cm,

- L: 46 x 43 x 8 cm,

- XL: 48 x 43 x 8 cm

AT03012: 34 x 40 x 8 cm

AT03013: 40 x 40 x 8 cm, - 37 x 37 x 8 cm

AT03014: length: 55 cm, width: 31 cm height: 7 to 11 cm

AT03015: length: 60 cm, width: 31 cm height: 7 to 11 cm

AT03016: 660 x 440 x 111 mm

AT03017: 420 x 330 x 111 mm

### CLEANING AND MAINTENANCE

#### Cleaning foam

Cleaning viscoelastic foam (also known as memory foam) requires a delicate approach to avoid damaging its structure. Here's a guide to properly clean it:

What You'll Need:

Mild detergent or dish soap

Warm water

Baking soda (optional for deodorizing)

Vacuum cleaner (preferably with a soft brush attachment)

Spray bottle

Soft cloth or sponge

Enzyme cleaner (optional for stains)

Steps for Cleaning:

#### Vacuum the Foam:

Remove dust and debris: Use a vacuum cleaner with a soft brush attachment to gently vacuum the surface of the foam. This will remove dust, dirt, and debris before you begin any wet cleaning.

#### Spot Clean with Mild Detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly mist the foam: Avoid soaking the foam. Lightly spray the mixture on the affected area.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not scrub, as this could damage the foam's structure.

Rinse the area: If needed, lightly spray with clean water to rinse off any soap residue.

#### Dealing with Tough Stains:

For urine or biological stains: Use an enzyme cleaner. Lightly apply the enzyme cleaner to the stain, let it sit for a while (following product instructions), then blot with a clean cloth.

Oil-based stains: Use a small amount of dish soap or a specialized upholstery cleaner designed for foam materials.

#### Deodorizing (Optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and let it sit for a few hours, or even overnight, to absorb odors.

Vacuum off the baking soda after it has sat for long enough.

#### Dry the Foam:

Air dry completely: Foam is very absorbent, so it can take time to dry. Leave the foam in a well-ventilated area.

Use a fan to speed up the process: Never use direct heat, such as a hair dryer, as this can damage the foam. Just a cool fan is ideal.

Ensure the foam is fully dry before using it again to prevent mold or mildew growth.

What to Avoid:

Never submerge the foam in water, as it can be difficult to dry and may cause damage.

Avoid using harsh chemicals like bleach or alcohol-based cleaners that can degrade the foam.

No direct heat: Heat can damage the structure of the memory foam, so avoid putting it in the dryer or using a hot air source to dry.

**Instructions for washing the cover** (models: AT03001, AT03002, AT03003, AT03004, AT03005, AT03006, AT03007, AT03008, AT03009, AT03010, AT03011, AT03012, AT03014, AT03015, AT03016, AT03017)

- 1) Hand wash only the cover.
- 2) Clean with a damp sponge and detergent.
- 3) Maximum wash temperature 40°C.
- 4) Do not chlorinate! (Do not bleach the cover).
- 5) Do not dry clean!
- 6) Do not tumble dry! Dry laid out flat in the shade.
- 7) Do not iron! (avoid contact with heated surfaces e.g. radiator)
- 8) Deterioration of visual-aesthetic properties (such as surface wrinkling) does not alter the performance of the material.

### Washing instructions for the disinfecting cover (AT03006D):

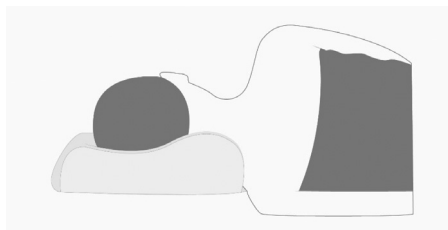
- 1) Clean using a damp sponge with detergent.
- 2) Maximum wash temperature 95 °C.
- 3) Do not chlorinate! (Do not bleach the cover).
- 4) Do not dry clean!
- 5) Do not tumble dry! Dry laid out flat in the shade.
- 6) Do not iron! (avoid contact with heated surfaces e.g. radiator).
- 7) Deterioration of visual-aesthetic properties (such as surface wrinkling) does not alter the performance of the material.

### STORAGE

The product should be stored in a dry cool room, away from moisture and direct sunlight.

### COMMENTS AND WARNINGS:

1. It is forbidden to use the product in children under 3 years of age. The soft structure of the sponge can seriously interfere with the breathing process when the baby is lying on his stomach.
2. Do not lie on the sleeping pillow in the prone position.
3. Avoid contact of the product with fire. The product may ignite.
4. The lumbar pillow should be placed on the backrest so that its flat part is in contact with the backrest, set at the height of the lumbar region.
5. The sleeping pillow (models: AT03001, AT03002, AT03007, AT03014, AT03015) should be placed so that the upper part of the pillow is under the neck, as shown in the picture below:

### CAUTION:

In the event of a product-related „serious incident“ that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above ‚serious incident‘ must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

### NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

**NOTE:** It is forbidden to use the product other than in accordance with its intended use

**ATTENTION:** The manufacturer is not responsible for damage caused by negligent cleaning, maintenance or as a result of not following the instructions in this manual.

### TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/ necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

### HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

Once the device has been withdrawn from service, the medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

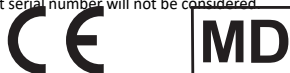
Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.

Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR MEDICAL Sp. z o.o.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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