AT03005

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage and personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION/INDICATIONS

- Cervical spine problems: People who suffer from neck pain or have cervical spine problems can use an orthopedic travel pillow to provide adequate support for the neck and head while traveling, especially on long car trips or flights.
 - Recovery from spinal surgery: After surgical procedures on the cervical spine, orthopedic pillows may be recommended by doctors to support the healing process and prevent excessive strain on the operated area.
 - Chronic neck pain: People suffering from chronic neck pain may find relief from orthopedic pillows, which provide better support and can help reduce muscle tension.
 - Improving sleep quality while traveling: Using an orthopedic pillow can also help improve the comfort and quality of sleep when traveling, which is important for frequent travelers who want to avoid fatigue and related health problems.
 - Preventing pain and discomfort: Regular use of an orthopedic travel pillow can help prevent problems that can result from sitting in one position for long periods of time, such as neck, shoulder and back pain.
- Using such a pillow is especially advisable if you are already experiencing discomfort or if you want to take preventive care of your health while traveling.

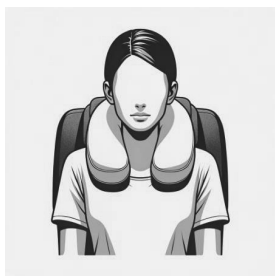
CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

- Back disorders: People with serious spinal problems, such as advanced osteoporosis, unstable fractures, or severe damage to intervertebral discs, should consult a doctor before using the orthopedic travel pillow, as certain positions may exacerbate these problems.
- Previous surgeries and injuries: If you have recently undergone surgery on the neck, spine or head area, the use of an orthopedic travel pillow may not be advisable without first consulting your doctor.
- Circulatory disorders: People with circulatory disorders may experience discomfort or worsened symptoms from using a pillow that puts too much pressure on the neck or interferes with blood flow.
- Dermal diseases: If you have active skin infections, eczema or other skin problems in the area where the pillow comes into contact with your skin, using it may lead to irritation or exacerbate the problem.
- Personal comfort: Some people may simply experience discomfort or pain when using this type of pillow because of its shape or firmness, which can negatively affect the quality of sleep or travel.

USE

Slip the pillow over your neck so that your head has stable support (as shown in the images below)

FEATURES

- made of dense visco-elastic foam with memory effect
- perfectly adapts to the shape of the body
- Dimensions: length 33 cm Width 33 cm Height 11 cm
- Available in four color variants: green, yellow, pink, blue.

COMPONENTS: orthopedic travel pillow, instruction manual.

NOTES:

In the event of a device-related “serious incident” that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or as a result of failure to follow the instructions in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the intended use of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

Cleaning viscoelastic foam (also known as memory foam) requires a delicate approach to avoid damaging its structure. Here’s a guide to properly clean it:

What You’ll Need:

- Mild detergent or dish soap
- Warm water
- Baking soda (optional for deodorizing)
- Vacuum cleaner (preferably with a soft brush attachment)
- Spray bottle
- Soft cloth or sponge
- Enzyme cleaner (optional for stains)

Steps for Cleaning:**Vacuum the Foam:**

Remove dust and debris: Use a vacuum cleaner with a soft brush attachment to gently vacuum the surface of the foam. This will remove dust, dirt, and debris before you begin any wet cleaning.

Spot Clean with Mild Detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly mist the foam: Avoid soaking the foam. Lightly spray the mixture on the affected area.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not scrub, as this could damage the foam's structure.

Rinse the area: If needed, lightly spray with clean water to rinse off any soap residue.

Dealing with Tough Stains:

For urine or biological stains: Use an enzyme cleaner. Lightly apply the enzyme cleaner to the stain, let it sit for a while (following product instructions), then blot with a clean cloth.

Oil-based stains: Use a small amount of dish soap or a specialized upholstery cleaner designed for foam materials.

Deodorizing (Optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and let it sit for a few hours, or even overnight, to absorb odors.

Vacuum off the baking soda after it has sat for long enough.

Dry the Foam:

Air dry completely: Foam is very absorbent, so it can take time to dry. Leave the foam in a well-ventilated area. Use a fan to speed up the process: Never use direct heat, such as a hair dryer, as this can damage the foam. Just a cool fan is ideal.

Ensure the foam is fully dry before using it again to prevent mold or mildew growth.

What to Avoid:

Never submerge the foam in water, as it can be difficult to dry and may cause damage.

Avoid using harsh chemicals like bleach or alcohol-based cleaners that can degrade the foam.

No direct heat: Heat can damage the structure of the memory foam, so avoid putting it in the dryer or using a hot air source to dry.

CLEANING THE COVER: unzip the cover with the zipper and remove the foam. The cover can be washed by hand at a maximum temperature of 40°C. Do not iron. Do not dry clean. Do not bleach. Do not tumble dry.

	Hand wash at up to 40°C
	Do not bleach
	Do not iron
	Do not tumble dry
	Do not dry clean

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight. Not suitable for children under three years of age. Store away from sources of fire.

HOW TO DISPOSE OF THE PRODUCT

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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