AT03018

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EN

Thank you for purchasing our product. Please read the user manual carefully. The product must not be used before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, please contact a healthcare specialist or the seller to avoid product damage or bodily injuries.

WARNING:

Check all parts for any damage that may have occurred during transport. If such damage is found, DO NOT use the product. More information is available from the seller.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment, rehabilitation, or compensation (see the section on product purpose in this manual). The product can be purchased independently by the user or based on the recommendations of a doctor, therapist, or another specialist.

Regardless of whether the product is purchased independently or recommended by a doctor/therapist/other specialist, the available sizes, necessary functions, and product variants, as well as indications and contraindications for use, should be considered, along with the information provided by the manufacturer.

APPLICATION/INDICATIONS

Made of innovative thermoforming memory foam, which perfectly adapts to the anatomical shapes of the body. The innovative foam prevents the development of microorganisms, allergens, and the absorption of dust particles. The pillow is used to prevent and alleviate the course of conditions and to relieve or compensate for knee injuries by keeping them separated while lying down.

When used in an unfolded form, it also serves as excellent support for the lower legs, ensuring proper patient positioning (e.g., after hip joint endoprosthesis) and for pressure ulcer prevention – preventing bedsores in the heel area.

PRODUCT FEATURES/PURPOSE

Helps ensure proper anatomical positioning of the pelvis and hips when lying on the side after surgery or during rehabilitation.

Effectively reduces pain caused by incorrect hip positioning when lying on the side.

Reduces tension in the hips and knee joints.

Provides ideal knee support, reducing pain caused by uncomfortable pressure.

The separator is made of memory foam, perfectly adapting to anatomical body shapes.

A soft-touch cover with a zipper facilitates keeping the separator clean.

Dimensions: 25x20x15 cm

Scope of delivery: Pillow, cover

Cover: Velour, navy blue

Pillow: Thermoforming foam

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product. The product is not suitable for children under 3 years old.

LIST OF COMPONENTS

Orthopedic separation pillow

User manual

USAGE

1. Place the pillow between the thighs (while lying on the side).
2. Place the pillow under the leg (while lying on the back).

WARNING:

If a "serious incident" related to the product occurs, which directly or indirectly led, could have led, or may lead to any of the following events:

- a) Death of the patient, user, or another person, or
- b) Temporary or permanent deterioration of the patient's, user's, or another person's health, or
- c) Serious public health threat,

this "serious incident" must be reported to the manufacturer and the competent authority of the member state in which the user or patient resides.

For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

If pain, allergic reactions, or other disturbing, unclear symptoms occur during product use, consult a healthcare professional.

WARNING:

The product must not be used in any way other than its intended purpose.

CLEANING

The cover should be washed gently at a temperature of up to 40°C.

DO NOT:

Iron

Tumble dry

Hang to dry

Bleach

Dry clean

Use strong detergents

Cleaning viscoelastic foam (also known as memory foam) requires a delicate approach to avoid damaging its structure.

What you will need:

Mild detergent or dishwashing liquid

Warm water

Baking soda (optional for deodorizing)

Vacuum cleaner (preferably with a soft brush attachment)

Spray bottle

Soft cloth or sponge

Enzymatic cleaner (optional for stain removal)

Cleaning steps:**Vacuuming the foam:**

Remove dust and dirt: Use a vacuum cleaner with a soft brush attachment to gently vacuum the foam surface. This removes dust, dirt, and debris before wet cleaning.

Spot cleaning with mild detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Lightly spray the foam: Avoid soaking the foam. Lightly spray the stained area.

Blot the stains: Use a soft cloth or sponge to gently blot the area. Do not rub, as this may damage the foam structure.

Rinse the area: If necessary, lightly spray with clean water to remove detergent residue.

Dealing with tough stains:

Biological stains or urine: Use an enzymatic cleaner. Apply the cleaner to the stain, let it sit for a while (as per product instructions), then blot with a clean cloth.

Grease stains: Use a small amount of dishwashing liquid or a special upholstery cleaner designed for foam materials.

Deodorization (optional):

Use baking soda: Sprinkle a layer of baking soda over the foam and leave it for several hours or overnight to absorb odors.

Vacuum the baking soda after enough time has passed.

Drying the foam:

Allow to air dry completely: Foam is highly absorbent, so it may take some time to dry. Leave it in a well-ventilated area.

Use a fan to speed up the process: Never use a direct heat source, such as a hairdryer, as this can damage the foam.

Ensure the foam is completely dry before reuse to prevent mold or mildew growth.

Avoid:

Never submerge the foam in water, as it is difficult to dry and may become damaged.

Avoid using strong chemicals like bleach or alcohol-based cleaners, which can weaken the foam structure.

No direct heat: Heat can damage the memory foam structure, so avoid tumble drying or using hot air.

STORAGE

Store the product in a dry place and do not expose it to direct sunlight.

DISPOSAL

Once decommissioned, the product should be treated as regular household waste. Follow waste segregation regulations. Protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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