AT51006C
INSTRUKCJA UŻYTKOWNIA, CHODZIK
INSTRUCTION MANUAL, WALKER
GEBRAUCHSANWEISUNG, GEHHILFE
NÁVOD K POUŽITÍ, CHODÍTKO
NÁVOD NA POUŽITIE, CHODÍTKO
MODE D'EMPLOI, DÉAMBULATEUR
GEBRUIKSAANWIJZING, LOOPREK
MANUAL DE INSTRUCCIONES, ANDADOR
MANUALE D'ISTRUZIONI, DEAMBULATORE
BRUKSANVISNING, ROLLATOR

MD

PL

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En

Thank you for purchasing our product. Please carefully read the user manual. Do not use the product before familiarizing yourself with this manual and understanding it. If you do not understand the warnings, notes, and recommendations, contact a healthcare specialist or the seller to avoid damage to the product or personal injury.

CAUTION: Check all parts for damage that may have occurred during transport. If such damage is noticed, DO NOT use the product. For more information, contact the seller.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for the treatment/rehabilitation or compensation of which this product is intended (see the section on product purpose in this user manual). The product can be purchased independently by the user or based on recommendations from a doctor, therapist, or other specialist. Whether purchased independently or recommended by a healthcare professional, available sizes/necessary functions, product variants, indications, contraindications for use, and information provided by the manufacturer should be considered.

GENERAL WARNING

DO NOT assemble or use the product before reading and understanding this manual. If you do not understand the warnings, cautions, or instructions, contact a healthcare specialist, distributor, or technical support before attempting to assemble the product to avoid injury or damage. Each user should consult their doctor or therapist to determine the appropriate adjustment variant and method of use. The walker is not designed for the user to steer it while sitting on it. When adjusting the walker's height, a doctor or therapist should be present to support the user and ensure proper brake settings and screw tightening. Always ensure that the seat tube is locked and properly attached to the frame, and that the brake cable is inside the side tube and securely fastened to it. After each height adjustment, check the brake cable to avoid loosening. Make sure the knobs are locked so that the main load-bearing structure does not shift under pressure. The quick-release mechanism is correctly mounted and will stay in the release position even under strong reverse pulling. Copper pins in the adjustable tube should protrude on the right side. Failure to follow these recommendations may result in injury. Before using, check that ALL adjustment elements of the handles and seat are secured and that the wheels and movable parts are in good condition. During use, ALL wheels must remain in contact with the ground at ALL TIMES to ensure stability. BEFORE sitting on the seat, the brakes MUST be in the locked position. When the walker is stationary, its hand brakes MUST be in the locked position.

WARNING! Improper use of the product may pose a risk of tipping over.

NOTE: Follow the recommendations for getting in/out/moving. After completing adjustments, stabilize the position by fully tightening the nuts/screws. The maximum load of the walker's bag is 5 kg.

WARNINGS ABOUT ASSEMBLY AND FOLDING

After unfolding or assembling the walker, ensure the seat is locked and both handle tubes are securely attached to the main frame. Verify that the copper pins protrude from the appropriate holes and are properly locked, and that both handles are at the same height and do not shift.

INSTALLATION WARNINGS

Ensure the seat tube is securely fastened in the lock, all copper pins protrude from the correct holes, and the front and rear tubes are safely inserted into the main frame and secured with knobs. ALWAYS check that the walker and its components are securely locked in place before use.

FOLDING/UNFOLDING THE WALKER

Place the walker on a flat surface, unfold the frame and wheels, then screw in the bolts to set up the front wheels as well as both tubes for the rear wheels (Fig. 1, 2, 3).

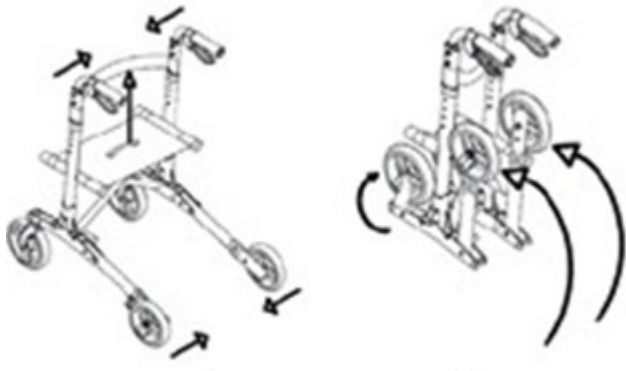
HANDLE TUBE ADJUSTMENT

Press the latch on the adjustment tube and slide it up or down to set the height. Ensure the latch is positioned in the appropriate hole after adjustment.

FOLDING

Lift the seat handle to fold the walker, then fold the wheels according to Fig. 4 and 5.

USING/LOCKING/UNLOCKING THE HAND BRAKES

CAUTION: During use, handling, folding, and adjusting mechanisms, there is a risk of pinching and/or squeezing the user's or caregiver's body parts in the openings/gaps between components. Perform these actions with great caution. After adjustment, stabilize the position by fully tightening the nuts/screws.

USING THE HAND BRAKES

Pull the brake towards the hand grips.

Perform one of the following:

A. **NOT MOVING:** Keep the brake handle pulled up.

B. **MOVING:** Release the brake handle.

LOCKING THE HAND BRAKES

Push the lower part of the brake handle down according to the illustration until you hear a click (Fig. 7).

UNLOCKING THE HAND BRAKES

Pull the brake handles up until released, as shown in Fig. 6.

6. Unlocked state

7. Locked state

INTENDED USE: The product is designed to support individuals with limited walking ability during mobility. The walker is designed for single-person use.

CONTRAINDICATIONS: Physical or mental limitations (e.g., impaired vision) that prevent safe handling of the walker.

FEATURES:

Lightweight aluminum design

Quick folding/unfolding capability

Equipped with an additional bag

Functions as a seat

Includes a cane attached to the walker, which can be detached and used as additional support, especially when getting up from the seat or moving through narrow spaces where using the full walker is difficult.

Maximum user weight: 110 kg.

MAINTENANCE

Ensure all components are always secured.

Replace damaged or worn parts immediately.

WHEEL MAINTENANCE

Clean the wheels with warm water and mild detergent. Dry with a cloth.

Periodically lubricate with 3-in-1 oil if the wheels start to squeak.

DISPOSAL

After decommissioning, the medical device can be disposed of as regular municipal waste.

TECHNICAL PARAMETERS:

Four-wheel aluminum walker, foldable.

Height: 79.5 - 92.5 cm

Seat height: 53.5 cm

Seat length: 46 cm

Seat width: 25 cm

Weight: 7.35 kg
Distance between handles: 46.5 cm
Distance between front wheels: 59 cm
Wheel diameter: front 8"
Distance between front and rear wheels: 51 cm
Maximum user weight: 136 kg

This symbol indicates the maximum weight of the user

COMPONENTS:

Walker, user manual.

NOTE:

In the event of a "serious incident" related to the product that directly or indirectly results in:

- a) death of the patient, user, or another person, or
- b) temporary or permanent health deterioration of the patient, user, or another person, or
- c) serious public health risk,

report the "serious incident" to the manufacturer and the competent authority of the member state where the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

In case of pain, allergic reactions, or other unclear symptoms related to using the medical device, consult a healthcare professional.

NOTE:

Using the product in ways other than its intended purpose is prohibited.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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