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## ENG

### INTENDED USE

The folding walker is designed for people with problems with verticalization, with diseases of the locomotor system or people with problems with maintaining balance while walking. The walker provides the user with support, reducing the risk of tipping over. The product will work well **also** when learning to walk or after injuries.

### CHARACTERISTICS

- made of reinforced aluminum
- equipped with a set of **anti-slip legs**
- easy to fold and transport, folded with one button
- 8 levels of height adjustment
- ergonomic foam grips

### CONTRAINDICATIONS

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

### WARNINGS:

1. Before using the walker, read this instruction manual and keep it for future reference.
2. Use of the product on ground that does not provide stability, i.e. wet, icy, greasy, very dusty ground, is not permitted **because of the risk** of falling.
3. Before each use, carry out a visual inspection of all structural components of the product including a thorough check of the stability and condition of the support frame.
4. The product is for support only and intended for one user. It is forbidden to stand on the walker.
5. Do not exceed the maximum height level.
6. Do not use if user weight exceeds 110 kg.
7. Do not use the walker if any visible or **palpable** damage is found (e.g. cracks). frame) as this may result in an accident.
8. Do not lean out of the walker. This may cause the user to fall over.
9. Do not swing the walker. This may cause the user to fall over.
10. Avoid contact with open fire.

### ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to, any of the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- c) a serious threat to public health

this "serious incident" should be reported to the manufacturer and to the competent authority of the Member State where the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

### ATTENTION:

In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of the medical device, consult a healthcare professional.

**ATTENTION:** it is forbidden to use the product in a way other than for its intended purpose!

**ATTENTION:** Before use, check that all elements have been assembled correctly.

**ATTENTION:** During the use and handling of the product, as well as during its assembly and adjustment of mechanisms, there may be a risk of entrapment and/or squeezing of the user's/assistant's body parts in the openings/gaps between the elements. These steps should be performed with particular care. After adjustment, stabilize the position by carefully tightening the nuts/bolts

**NOTE:** The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

#### TECHNICAL PARAMETERS

Height: 72 - 89.5, 8 degrees of adjustment, every 2.5 cm

Width between handles: 49 cm

Width: 60cm

Folded width: 11 cm

Length: 51

Packaging dimensions: 580x120x760 cm

Maximum user weight: 110 kg

Weight: 2.3 kg

Weight with carton: 3.3 kg

THIS MARK INDICATES THE MAXIMUM USER WEIGHT.

#### UNFOLDING

To unfold the walker, pull the sides of the walker outwards.

#### FOLDING

To fold the walker, press the latch button and pull the sides of the walker inwards, and then fold the walker.

#### MOVING BY THE WALKER WITH THE STEPPING FUNCTION ENABLED

The stepping function is turned on when the locking pins of the stepping function of the walker are in the front reinforcement. Stand by the walker, holding the handles of the walker with both hands. Moving with this type of equipment consists in alternately lifting the sides of the walker up and moving them forward and **then moving forward** the lower limbs.

#### MOVING BY THE WALKER WITH LOCKED FUNCTION ON

Moving with a fixed walker consists in moving it forward and **then moving forward** the lower limbs. The walking function lock should be in the side holes.

#### TARGET GROUP OF PATIENTS

People suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer.

#### PRODUCT MAINTENANCE

The product should be periodically cleaned with water and mild detergents. The product may be subjected to action disinfectants. Make sure that the disinfectant does not contain harsh chemicals such as strong acids and alkalis. Do not use rough cleaning objects or strong detergents.

#### STORAGE

When not in use, the product should be stored in a dry and cool place in its original packaging. Avoid damp rooms and prolonged exposure to direct sunlight.

#### METHOD OF DISPOSAL THE PRODUCT

The medical device can be disposed of as normal household waste.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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