AT51042
BALKONIK INSTRUKCJA OBSŁUGI
BALCONY MANUAL
BALKON-HANDBUCH
NÁVOD NA BALKON
NÁVOD NA BALKÓN
BALCON ET MANUEL D'INSTRUCTIONS
BALKON EN INSTRUCTIEHANDLEIDING
BALCÓN Y MANUAL DE INSTRUCCIONES
BALCONE E MANUALE DI ISTRUZIONI
BALKONG OCH BRUKSANVISNING

MD

PL

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ENG

PURPOSE

The folding balancer is intended for persons with problems with standing up, with locomotor system disorders or persons with problems with maintaining balance while walking. The balcony provides the user with support, reducing the risk of falling over. The product is useful when learning to walk or after injuries.

FEATURES

- made of reinforced aluminium
- 4-inch front swivel and 4-inch rear wheels.
- easy to fold and transport with the push of a button
- 8 steps of height adjustment
- ergonomic PVC handles

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

WARNINGS:

1. read this user manual before using the balcony and keep it for future reference.
- 2) Do not use the stool on unsupportive surfaces, i.e. wet, icy, greasy or very dusty ground.
Before each use, - make a visual inspection of all structural components of the product including: - carefully check the stability and condition of the supporting frame
4. the balcony is only to be used to support one user, it is forbidden to stand on the balcony.
5. do not exceed the maximum height level.
6. do not use if the weight of the user exceeds 110 kg
Do not use the stool if there is any visible or perceptible damage (e.g. a crack in the frame) as this can cause an accident.
7. Do not use the stool if there is any visible or tactile damage (e.g. breakage of the frame).
8. Do not lean over the balcony. This can cause the user to fall over.
Do not rock the stool. This can cause the user to fall over.
10. avoid contact with open flames

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

CAUTION:

If you experience pain, allergic reactions or other disturbing symptoms that are unclear to you related to the use of the medical device, consult your healthcare professional.

ATTENTION: It is forbidden to use the product other than for its intended purpose!

NOTE: Before use, check that all components have been assembled correctly.

ATTENTION: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger of entrapment and/or compression of parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Stabilise the position by tightening the nuts/screws firmly after adjustment.

ATTENTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of failing to observe the recommendations contained in this user manual.

TECHNICAL SPECIFICATIONS

Height: 85-102.5 cm, 8 degrees of adjustment, in 2.5 cm increments

Width between handles: 42 cm

Width: 63 cm

Width when folded: 12 cm

Length: 48 cm

Packaging dimensions: 56x12x75 cm

Solid wheels: 4"

Maximum user weight: 110 kg

Weight: 3.75 kg

Weight with carton: 4.5 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER.

UNFOLDING

To unfold the balcony, pull the sides of the balcony outwards and then fit the legs to the correct height.

FOLDING

To fold the balcony, press the latch button and pull the sides of the balcony inwards, then fold the balcony.

MOVING THE BALCONY

Stand at the balcony holding the handles of the balcony with both hands, the angle of flexion of the legs should be approximately 30°. To stop the stool, put your body weight on the handles.

TARGET PATIENT GROUP

The healthcare professional indicates the use of the product for adults and children on their own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

MAINTENANCE OF THE PRODUCT

The product should be cleaned periodically using water and mild cleaning agents. The product may be treated with

disinfectants. Ensure that the disinfectant does not contain corrosive chemicals such as strong acids and alkalis. Do not use rough objects for cleaning or strong detergents.

STORAGE

When not in use, the product should be stored in a dry and cool room in its original packaging. Avoid damp rooms and prolonged exposure to direct sunlight.

HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE

After the device has been taken out of service, the medical device can be disposed of as normal municipal waste with the exception of electrical products - proceed in a manner appropriate for the disposal of electrical and electronic equipment.

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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