AT51044

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INSTRUCTION MANUAL, ROLLATOR

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

USE/INDICATIONS

The product is designed to support the movement of people with limited walking ability. The rehabilitation rollator is designed for one person. The product can be applied in the following areas:

- Balance Training: The rehabilitation rollator allows patients to practice maintaining balance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use a rehabilitation rollator to train correct posture, which is important especially for those with postural defects or problems maintaining body stability.
- Improving motor coordination: Exercises performed with a rehabilitation rollator can help improve motor coordination and movement control, which is important for people with neurological disorders.
- Supporting rehabilitation after injuries: The rehabilitation rollator can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rehabilitation rollator can be a tool to increase independence in daily life, enabling independent movement.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- Lack of spinal stability: People with severe spinal stability problems or spinal injuries may not be suitable candidates for using a rehabilitation rollator, as it may increase the risk of further injury.
- Uncontrolled seizures: In people with uncontrolled seizures or epilepsy, the use of a rehabilitation rollator can be dangerous for both the patient and those around them.
- Poor muscle strength: In people with very limited muscle strength, using a rehabilitation rollator may be difficult or ineffective. In such cases, a different approach to rehabilitation may be necessary, such as manual therapy or Muscle Strength Support Exercises.
- Balance problems: In patients with severe balance problems, the use of a rollator can be dangerous because it can increase the risk of falls and further injury.
- Rehabilitation rollator mismatch: Improperly fitting a rehabilitation rollator to a patient can lead to additional injury or worsening of the condition.

WARNINGS:

- 1 If you are not using the rollator, check that the parking/parking brakes are on.
- 2 Before use, check the general condition of the wheels and ensure they are well mounted.
- 3 Check that the brakes work before use.
- 4 Make sure the rollator is fully extended.
- 5 When using a rollator pay attention to your surroundings to move safely
- 6 Do not use the rollator as a wheelchair. The product is designed to assist the walking process.
- 7 Do not sit on the rollator if the parking brakes are not engaged, as you risk moving the rollator, which will lead to danger.
- 8 Sit on the seat carefully, slowly. If necessary, ask a third party for help.
- 9 Check that the adjustable handles are properly secured.
- 10 Do not put your hands where the rollator folds to avoid injury.
- 11 Do not hang any objects on the handles or brakes.

FEATURES

DIMENSIONS:

WIDTH: +-61 cm

DEPTH: +- 75 cm

SEAT SIZE: 46 cm X 25 cm

SEAT HEIGHT: +- 55 cm

HEIGHT ADJUSTMENT OF HANDLES: 84-94 cm(5 degrees of adjustment)

MAXIMUM USER WEIGHT: 136 kg

FRONT WHEELS: 10 inches

REAR WHEELS: 8 inches

WEIGHT: +- 7.7 kg

SCOPE OF DELIVERY:

Rollators frame, front wheels, rear wheels, bag, backrest, cane holder, wheel mounting knobs, instruction manual.

MONTAGE


Remove the parts from the cardboard box.

Be careful not to damage the product when unpacking.

INSTALLATION OF FRONT WHEELS

Insert the front wheel into the frame, press the button to assemble it.

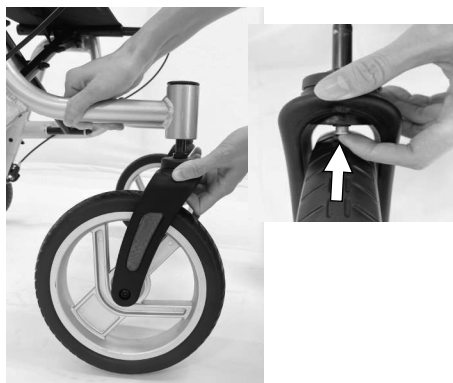**INSTALLATION OF REAR WHEELS**

1) To insert the rear tube, make sure the holes centrally overlap

2) Insert the screw from the inside and apply the black washer, then tighten the five-star knob.

3) Make sure the knob is tightened

BRAKE ADJUSTMENT

Turn the adjustment screw counterclockwise to increase the tension, and then lock the adjustment nut clockwise while holding the adjustment screw stationary.

BACKREST ASSEMBLY

Slide the backrest into the designated holes(as shown in the picture below)

CANE HOLDER ATTACHMENT


INSTALLATION OF A SHOPPING BAG

HEIGHT ADJUSTMENT OF HANDLES


Unscrew the orange knob, then set the height that suits you. Finally, turn the orange knob.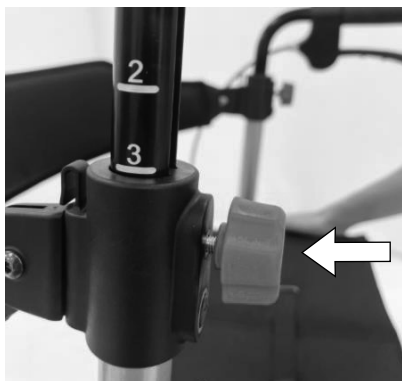


SETTING PARKING/PARKING BRAKES

To apply the parking brake, push the brake handles down.

To release the parking brakes, grasp the brake handles again and pull them upward until they release.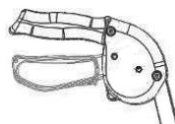

(unlocked state)

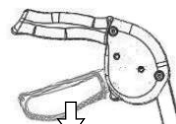(locked state)

USE OF HANDBRAKES

1. Pull the brake in the direction of the hand grips.

2. Perform one of the following actions:

A. NOT MOVING - hold the brake handle up:

B. MOVING - release the brake handle.

NOTES:

Do not stand on the product. The handles should be adjusted so that they are at the same height.

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

(a) death of a patient, user or other person or

(b) temporary or permanent deterioration of the health of the patient, user or other person or

(c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

Remember to check your rollator regularly. Before each use, check that all screws and knobs are tightened. Check that the brakes are working properly. Check the wear condition of the wheels. Check the looseness of the brake cable and the strength of the cable. Check

Whether any components are missing. If you find any defects or malfunctions, do not use the rollator, as it risks danger.

The rollator can be cleaned with a damp scrubber and mild soap. If necessary, you can use mild disinfectants and antibacterial agents. Do not rub with rough objects.

Be careful not to leave any cleaning residue on the handles or seat, as this can make these items slippery during use.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

METHOD OF DISPOSAL OF THE PRODUCT

Once the product has been taken out of service, the medical device can be disposed of as normal municipal waste. If necessary, contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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