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EN

Thank you for purchasing our product. Please read this user manual carefully before use. The product must not be used before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or distributor to avoid damage to the product or injury.

**WARNING:** Please inspect all parts for damage caused during transport. If any such damage is found, DO NOT use the product. Contact your distributor for more information.

#### **TARGET USER GROUP**

Individuals affected by conditions, dysfunctions, or injuries for which this device is intended (see the "Intended Use" section of this manual). The product may be purchased by the user independently or based on recommendations from a doctor, therapist, or other specialist. Whether purchased independently or upon recommendation, the following must be considered: available sizes/necessary features and versions, indications and contraindications for use, and the information provided by the manufacturer.

#### **INDICATIONS**

This product is intended to assist in the movement of individuals with limited walking ability. The rehabilitation walker is designed for use by one person. It is suitable in the following cases:

The walker allows patients to practice balance, which is important in rehabilitation following injuries or neurological disorders.

Patients can use the walker to train correct posture, particularly beneficial for those with postural defects or issues with body stability.

Exercises performed using the walker may help improve motor coordination and movement control, which is vital for individuals with neurological impairments.

The walker can be used in rehabilitation after lower limb injuries, helping patients gradually regain mobility.

For some individuals, the walker can be a tool to increase independence in daily life by enabling self-movement.

#### **CONTRAINDICATIONS**

Physical or mental limitations (e.g. vision impairment) that prevent safe use of the product.

Individuals with serious spinal instability or injuries may not be suitable candidates, as the walker may increase the risk of further injury.

People with uncontrolled seizures or epilepsy may pose a danger to themselves or others when using the walker.

Those with severely limited muscle strength may find the walker difficult or ineffective to use. Alternative rehabilitation approaches may be needed, such as manual therapy or muscle-strengthening exercises.

In cases of severe balance disorders, the use of a walker may increase the risk of falls and further injuries.

#### **LIST OF COMPONENTS**

Walker frame, front wheels, shopping bag, crutch holders, user manual

#### **FEATURES**

The rehabilitation walker frame is equipped with four solid wheels. The two front wheels have adjustable swivel forks with quick-release function, ensuring high comfort and ergonomic use.

The walker includes brakes with a locking function, a seat, a removable backrest, and a shopping bag.

#### **TECHNICAL SPECIFICATIONS**

Frame: Aluminium

Wheel fork: Plastic (PA)

Bag: Woven fabric

Seat: Foam and woven fabric

Backrest: Nylon

Handle: TPR

Dimensions:

Total width: 63.5 cm

Total depth: 74.5 cm

Seat width: 46 cm

Seat depth: 25 cm

Seat height: 55 cm

Distance between handles: 50 cm

Folded walker dimensions: Length 81 cm, width 28 cm, height 39 cm

Adjustable handle height: 86.5–96.5 cm

Front wheels: 10-inch TPU

Rear wheels: 8-inch TPU

Maximum user weight: 136 kg

Weight: 9.8 kg

This symbol indicates the maximum user weight

NOTE! Product dimensions may vary by  $\pm 2$  cm.

#### **WARNING:**

In the event of a "serious incident" related to the product, which has led, could have led, or may lead directly or indirectly to any of the following:

- a) the death of a patient, user, or another person;
  - b) a temporary or permanent deterioration in the health of a patient, user, or another person;
  - c) a serious threat to public health –
- this “serious incident” must be reported to the manufacturer and the competent authority of the Member State in which the user or patient resides.

**NOTE:**

If you experience pain, allergic reactions, or other disturbing and unclear symptoms while using the medical device, consult a healthcare professional.

**WARNING:**

It is prohibited to use the product in any way other than its intended purpose.

**SAFETY WARNINGS**

**Walker unfolding:**

Ensure the walker is fully unfolded and the seat is in the lowered position.

**Handle height:**

Make sure the handles are adjusted to the same height.

**Brake test:**

Test the brakes before using the walker. When the brakes are locked, the rear wheels should not turn. If they do, return the device to the seller for adjustment.

When the brakes are released, the wheels should rotate freely.

**Locking brakes before sitting:**

Always lock the brakes before sitting down. Do not sit on the seat if the device is on an inclined or uneven surface.

**Wheel replacement:**

Replace wheels if they are significantly worn or damaged.

**Positioning before sitting/standing:**

Before sitting on or getting up from the seat, lock both rear wheels using the double locks. Ensure the front wheels are pointing forward.

**Avoid leaning forward:**

Leaning forward while using the walker as a transport chair without assistance may cause tipping.

**Extreme temperatures:**

Do not expose the product to temperatures above 40°C (104°F) for extended periods, as this may cause overheating of walker components and burn the user.

**Folding bar lock:**

Ensure the walker is fully opened, the seat lowered, and the folding bar under the seat is locked in the downward position.

**Device stability:**

Make sure both adjustment knobs are fully tightened and at the same height. Test the stability of the walker before leaning on the handles.

**Regular inspection:**

Check all screws and fasteners regularly and tighten if necessary.

**Not for use in moving vehicles:**

Do not use the device in moving vehicles.

**Exiting the transport chair:**

Lock the wheels by pressing the brake handles.

Remove feet from footrests and place them firmly on the ground.

Use the armrests for support when standing up. Assistance from another person is recommended.

**Storage bag:**

Use the bag only to carry items. Do not overload it.

Maximum load: 5 kg

**WALKER ASSEMBLY**

**Front wheel installation:**

Insert the front wheel into the designated hole.

Ensure you hear the red safety latch click, confirming the wheel is securely attached.

Repeat for the second wheel.

**Frame unfolding:**

Lift the handle and unfold the frame by pressing down on both sides of the seat until stable.

If the handles do not click, the frame is not fully unfolded and secured.

#### Armrest height adjustment:

Press the adjustment button and set the desired height for both armrests.  
Ensure both sides are set to the same height.

#### HANDLE HEIGHT ADJUSTMENT

##### Position Locking:

After setting the desired armrest height, release the button to secure the position.

##### Even Height:

Ensure that both handles are set to the same height.

##### Proper User Posture:

The handles should be adjusted so that when the user is standing upright, the elbows remain slightly and naturally bent.

The arms should rest in a relaxed, natural position.

The elbows should not be fully extended or overly raised.

Avoid leaning, hunching, or excessive shoulder and scapular extension.

##### Optimal Height:

For the best fit, always consult a doctor or physical therapist.

#### BRAKE CHECK AND USAGE

##### Brake Test Before Use:

Press the brake lever down. The walker should gradually slow and come to a complete stop.

When fully pressed down, the brake should engage and immobilize the device.

##### Releasing the Brakes:

To release the brakes, lift the lever upward.

##### Simultaneous Use of Both Brakes:

Always use both brakes at the same time to ensure stability and safety while stopping.

##### Warning:

While walking, always keep your hands ready on the brake lever to stop quickly if needed.

#### BRAKE ADJUSTMENT

##### Wheel Functionality Check:

Wheels should spin freely and not rub against anything.

With the brakes fully engaged, the wheels should not move.

#### Brake Cable Tension Adjustment:

If the above conditions are not met, use the adjustment screws to correct the cable tension. Turn the screw clockwise to increase tension.

#### Post-Adjustment Test:

After adjustment, release the brakes and check that the wheels spin freely.

With the brakes engaged, ensure the wheels remain immobile.

#### Important Note:

If the brake is too loose, it may impair braking performance, reducing user safety.

Ensure the cable tension is appropriate for effective braking.

#### **SAFETY CHECK BEFORE USE**

##### Component and Mounting Inspection:

Ensure all parts and fasteners are securely in place.

Check that components are in good condition and not worn, torn, frayed, or loose.

##### Ensure Safe Use:

Make sure there are no obstacles or damage that could hinder normal and safe use.

##### Avoid Using a Damaged Product:

If any part or mechanism appears faulty or damaged, do not use the product.

#### **CLEANING**

All walker surfaces can be cleaned with a cloth and clean water, with a small amount of mild detergent if necessary. Then wipe with a dry cloth.

Do not use strong detergents or rough cloths/materials, as this may scratch the walker's surfaces.

Before each use, check the walker's condition.

If the wheels come into regular contact with wires, hair, or other debris, they may become blocked.

Check regularly to ensure the wheels rotate freely and are in good condition.

Frame (deformation, stability, joints).

Handles (secure height adjustment locking, proper support).

Seat (wear or damage).

To ensure long-term and safe use of the product, systematically inspect all screws and mounting components.

Before each use, check that all screws are properly tightened and show no signs of loosening.

Regularly check for missing or worn parts and ensure all components are in good working order.

If you notice wear, mechanical damage, or missing parts, stop using the product immediately and contact the manufacturer's service department to avoid accidents or further damage.

Pay attention to any unusual sounds or vibrations, which may indicate loose parts or other technical issues.

Always use original parts recommended by the manufacturer for replacements.

#### **STORAGE**

Store the product in a cool, dry place, away from moisture and direct sunlight.

#### **DISPOSAL OF THE PRODUCT**

Once withdrawn from use, the medical device can be disposed of as regular household waste.

Be sure to follow local waste separation regulations. Protect the environment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

Date of issue of the manual: 18.12.2024  
v1-18.12.2024