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EN

Thank you for purchasing our product. Please read the user manual carefully before use. Do not use the device before reading and fully understanding this manual. If you do not understand the warnings, cautions, or recommendations, please consult a healthcare professional or your vendor to avoid product damage or personal injury.

WARNING:

Inspect all parts for damage that may have occurred during transport. If such damage is found, DO NOT use the product. Contact the seller for further information.

INTENDED PATIENT GROUP

This device is intended for individuals with diseases, dysfunctions, or injuries requiring treatment, rehabilitation, or compensation (see the "Intended Use" section of this manual). The product may be purchased independently by the user or based on recommendations from a physician, therapist, or other specialist. Whether purchased independently or recommended by a specialist, the following should be considered: available sizes, required functions and variants of the device, indications and contraindications for use, and information provided by the manufacturer.

PURPOSE/APPLICATION / INDICATIONS

Product description:

The rollator is a modern walking aid designed for individuals with limited mobility. It is suitable for both indoor and outdoor use, providing comfort and safety while walking.

The rollator is intended for individuals who:

- Have weakened lower limb muscle strength,
- Experience balance and coordination issues,
- Are in recovery after injuries or orthopedic/neurological surgeries,
- Suffer from chronic musculoskeletal disorders (e.g., osteoarthritis, rheumatoid arthritis),
- Are elderly and require support while walking.

Applications:

- Support with walking indoors and outdoors,
- Offloading of lower limbs,
- Increased user safety and independence,
- Opportunity to rest thanks to the integrated seat and backrest,
- Facilitates the transport of small items using the attached bag.

CONTRAINDICATIONS

- Physical or mental limitations (e.g., visual impairment) preventing safe handling of the device,
- Complete inability to maintain an upright standing position independently,
- Inability to operate the brakes or lack of control over the device,
- Severe balance or orientation disorders (e.g., advanced dementia, severe ataxia),
- User weight exceeding the maximum load capacity (136 kg),
- Use of the rollator for unintended purposes (e.g., as a transport chair),
- Use on uneven, steep, or slippery surfaces without proper supervision.

LIST OF COMPONENTS: Rollator, backrest, crutch holder, cup holder, user manual

1. Handle
2. Adjustable handle tube
3. Brake lever
4. Handle lock button
5. Backrest
6. Seat
7. Bag hook
8. Front wheel
9. Brake adjustment screw
10. Rear wheel
11. Safety latch
12. Curb assist step
13. Crutch holder

FEATURES

- Foldable design – easy to store and transport; takes up minimal space when folded.
- Detachable rear legs – quick removal and attachment for convenient transport.
- Adjustable handle height.
- Brake with locking function.
- Crutch or cane holder.
- Integrated curb assist – enables easy navigation over thresholds and obstacles.
- Seat with backrest – allows safe and comfortable resting during walks.
- Shopping bag – hangs on a hook beneath the seat, maximum load 5 kg.
- Cup holder.

TECHNICAL SPECIFICATIONS

Weight: 7.35 kg
Dimensions (unfolded): 66 × 54 × 100 cm
Dimensions (folded): 66 × 24 × 87 cm
Seat height from ground: 60 cm
Seat dimensions: 43 × 20 cm
Handle height adjustment range: 87 cm – 100 cm
Maximum width: 54 cm
Maximum user weight: 136 kg
Maximum bag load: 5 kg
Maximum operating temperature: up to 40°C (104°F)
Seat and bag material: textile fabric
WARNING! Individual product dimensions may vary within ±2 cm.

This symbol indicates the maximum user weight.

DETACHABLE REAR LEGS

Thanks to the detachable rear legs, the device can be packed compactly and easily for transport. Simply press the button located under the rear tube to release the rear legs. To reattach them before use, follow the same procedure.

UNFOLDING THE ROLLATOR

Open the safety latch and spread the handles apart. Push the side bars downward until you hear a click—this indicates the device is fully unfolded. Before use, ensure that the folding mechanism is securely locked.

HANDLE HEIGHT ADJUSTMENT

Lift the locking clip and slide the handle up or down to the desired height. Then press the clip down—the handle will lock into the selected position.

ATTACHING / REMOVING THE BAG

To attach the bag strap, hook it onto the bag hook located at the front of the beam beneath the seat, facing outward.

BRAKING / LOCKING

Pull both brake levers upward to brake while moving with the rollator.

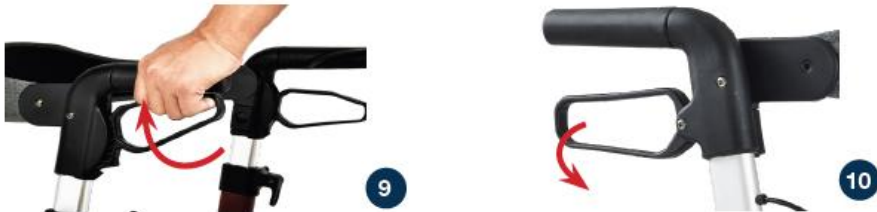
Always brake slowly and smoothly.

Push both brake levers downward to lock the wheels when the rollator is stationary.

To release the lock, pull the levers upward and release.

Brake performance may decrease due to wheel wear.

Regularly check the function of the brakes.

BRAKE ADJUSTMENT

To adjust braking strength, pull the brake cable upward, then set the desired tension by sliding the nut up or down.

SEATING

Place the rollator on a flat, stable surface. Lock the brakes (parking position).

Ensure the rollator stands firmly and does not slip.

Sit upright and do not lean backward.

DO NOT MOVE THE ROLLATOR WHILE SEATED.

FOLDING / STORAGE

To fold the rollator, pull the seat upward using the handle located on the seat plate. The rollator can now be fully folded and secured with the safety latch.

CURB SUPPORT

Use the integrated curb support when needed.

It allows you to overcome obstacles such as thresholds and curbs without lifting the rollator.

Simply press the support with your foot and gently pull the handles toward you—the front wheels will lift, making it easier to maneuver over the obstacle.

CRUTCH HOLDER

The stylish crutch holder consists of two parts: an upper hook and a lower foldable base. It is designed to carry a crutch or cane together with the rollator.

IMPORTANT:

In the event of a “serious incident” related to the product that has led, might have led, or could lead to any of the following:

- a) the death of a patient, user, or another person,
- b) a temporary or permanent serious deterioration in their health condition, or
- c) a serious risk to public health,

this “serious incident” must be reported to the manufacturer and the competent authority of the EU Member State where the user or patient resides.

WARNING: If pain, allergic reactions, or other unclear or concerning symptoms occur while using this medical device, consult a healthcare professional. **WARNING:** Using the product in any way other than its intended purpose is strictly prohibited.

NOTICES AND WARNINGS

Do not use the rollator to transport people or push it at high speed.

The device is not intended to carry passengers — such use may lead to tipping over and injury.

Do not leave the rollator outdoors for an extended period.

Moisture, intense sunlight, and extreme temperatures can damage metal, plastic, and textile components.

Do not use the rollator on steep, slippery, or highly uneven surfaces without supervision.

This may cause the rollator to tip over and the user to fall.

Do not overload the bag — the maximum load is 5 kg.

Excess weight may cause instability and tipping of the rollator.

Do not exceed the maximum user weight capacity of 136 kg.

Overloading can lead to permanent structural damage.

Always engage the brakes before sitting down or standing up from the seat.

This prevents the rollator from moving unexpectedly.

Do not sit on the rollator while it is in motion.

The seat is intended only for resting when the rollator is stationary and brakes are engaged.

CLEANING

1. General Cleaning

Clean the frame, handles, seat, and wheels regularly, especially after intensive outdoor use.

Use a soft, damp cloth and mild cleaning agents such as liquid soap or gentle dishwashing detergent.

Avoid harsh chemicals, solvents, alcohol, bleach, or abrasive products that could damage the surface of the frame or plastic components.

After cleaning, dry all parts with a dry cloth, especially metal and moving parts, to prevent corrosion.

2. Cleaning Textile Parts and Bag

If the shopping bag or other textile accessories become dirty, clean them by hand.

Do not dry on a radiator or in a tumble dryer — air drying at room temperature is recommended.

3. Wheel Cleaning and Inspection

Remove sand, mud, stones, hair, and other debris from the wheels that could impair rotation or braking.

If needed, wash the wheels with warm water and mild detergent.

Check the condition of the tires regularly — if they are heavily worn or cracked, they should be replaced.

4. Maintenance of Moving Parts and Brakes

Every 1–2 months (or more frequently with heavy use):

Check the brake performance (ensure wheels are properly locked when brakes are engaged),

Inspect the brake cable tension and adjust it using the adjustment screw at the wheel if necessary,

Ensure all bolts and nuts are properly tightened.

STORAGE

Store the product in a dry place. Do not expose it to direct sunlight.

DISPOSAL

When taken out of service, the product should be treated as standard household waste.

Be sure to follow local waste sorting regulations. Take care of the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

Date of issue of the manual: 04.02.2025
v1-04.02.2025