AT51055
INSTRUKCJA UŻYTKOWNIA, CHODZIK
INSTRUCTION MANUAL, ROLLATOR
GEBRAUCHSANWEISUNG, ROLLATOR
NÁVOD K POUŽITÍ, CHODÍTKO
NÁVOD NA POUŽITIE, CHODÍTKO
MANUEL D'INSTRUCTION, DÉAMBULATEUR
GEBRUIKSAANWIJZING, ROLLATOR
MANUAL DE INSTRUCCIONES, ANDADOR
MANUALE DI ISTRUZIONI, DEAMBULATORE
BRUKSANVISNING, RULLATOR

PL

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En

Thank you for purchasing our product. Please read the user manual carefully before use. Do not operate the rollator before reading and fully understanding this instruction manual. If you do not understand the warnings, cautions, or recommendations, consult a healthcare professional or the seller to avoid damage to the product or injury. **WARNING:** Check all components for any damage that may have occurred during transport. If such damage is noticed, DO NOT use the product. Contact the seller for more information.

INTENDED USER GROUP

The rollator is intended for individuals with conditions, dysfunctions, or injuries requiring treatment, rehabilitation, or compensation (see "intended use" section of this manual).

The product may be purchased independently by the user or based on a recommendation from a physician, therapist, or other healthcare professional.

In both cases, the user must consider available sizes, required features, model variants, usage indications and contraindications, and the manufacturer's instructions.

INDICATIONS / APPLICATION

The AT51055 is a foldable four-wheeled rollator with brakes, a seat, and backrest.

It provides stable support while walking and allows for resting.

It is designed for indoor and outdoor use by individuals with limited mobility.

Indications for use:

The rollator is suitable for individuals who:

Have weakened lower limb muscle strength

Experience balance or coordination disorders

Are recovering from injuries or orthopedic/neurological surgeries

Suffer from chronic musculoskeletal conditions (e.g., osteoarthritis, rheumatoid arthritis)

Are elderly and require support while walking

Applications:

Support with walking indoors and outdoors

Offloading pressure from the lower limbs

Enhancing user safety and independence

Providing rest with the integrated seat and backrest

Facilitating transport of small items in the included bag

CONTRAINDICATIONS

Physical or mental limitations (e.g., impaired vision) that prevent safe use

Complete inability to maintain a standing position independently

Inability to operate the brakes or control the rollator

Significant balance or orientation impairments (e.g., severe dementia, ataxia)

User weight exceeding the maximum limit of 136 kg

Improper use of the rollator (e.g., as a wheelchair to carry a person)

Use on uneven, steep, or slippery surfaces without assistance

COMPONENT LIST

1. Handles 2. Brakes 3. Backrest 4. Seat 5. Bag 6. Wheels

FEATURES

Foldable 4-wheel rollator for indoor and outdoor rehabilitation use

Lightweight and durable aluminum frame

Comfortable seat with backrest for resting during walks

Intuitive brake system with parking lock function

Height-adjustable handles to suit user height

Shopping bag mounted under the seat

Additional accessories: cane holder, cup holder, curb assist pedal
Safety latch to lock the rollator in folded position for transport
Durable wheels suitable for asphalt and indoor flooring

TECHNICAL SPECIFICATIONS

Frame material: Aluminum

Unfolded dimensions: 64 cm (L) × 61 cm (W) × 95 cm (H)

Folded dimensions: 64 cm (L) × 24 cm (W) × 82 cm (H)

Seat size: 46 × 38 × 20 cm

Seat height: 52 cm

Note: Actual product dimensions may vary by ±2 cm.

This symbol indicates the maximum user weight.

Handle height: 85.5–98 cm (6 adjustment levels, 25 mm intervals)

Maximum user weight: 136 kg

Weight: 7.9 kg

Maximum bag load: 5 kg

SUPPLIED COMPONENTS

Rollator, backrest, cane holder, cup holder, tray, user manual

UNFOLDING THE ROLLATOR

Open the safety latch and pull the handles outward.

Press the side frames downward until you hear a click — this confirms the rollator is fully opened.

Before use, ensure the folding mechanism is securely locked.

REMOVING THE REAR LEGS

The rear legs are detachable to facilitate transport.

To remove them, press the button located under the tube.

To reattach, repeat the same steps in reverse order.

ATTACHING / DETACHING THE BAG

Hook the bag strap onto the hook located at the front of the seat.

BRAKING AND WHEEL LOCKING

Pull the brake levers upward to brake while moving. Push the levers downward to lock the wheels when stationary. Release the lock by pulling the levers upward again. Regularly check brake performance.

ADJUSTING HANDLE HEIGHT

Lift the locking mechanism and position the handle at the desired height. Push the lock downward to secure it.

ADJUSTING BRAKING FORCE

Lift the brake cable. Adjust the tension by moving the adjustment screw up or down.

SEATING

Place the rollator on a flat, stable surface. Engage the brakes. Sit upright without leaning backward. Note: Do not move the rollator while seated.

FOLDING AND STORAGE

Pull the seat upward using the handle. Secure the folded rollator using the safety latch.

CURB ASSIST FUNCTION

Use the integrated curb support.

Press the foot on the assist pedal and gently pull the handles toward you – the front wheels will lift, making it easier to pass over curbs or obstacles.

CANE HOLDER

Consists of two parts – an upper hook and a lower foldable base – designed for transporting a cane or walking stick.

CUP HOLDER

Allows safe and convenient transport of beverages.

TECHNICAL SPECIFICATION OF THE ROLLATOR TRAY

Dimensions: 42 × 30 × 7 cm

Maximum load: 5 kg

Weight: 0.4 kg


ROLLATOR TRAY ASSEMBLY INSTRUCTIONS

Assembly of Parts

Please assemble the parts in the order from “1” to “5” shown in the diagram.

After tightening screw “1”, install latches “2” and “5”.


Please pay attention to the scale and the L/R markings to distinguish the left and right sides.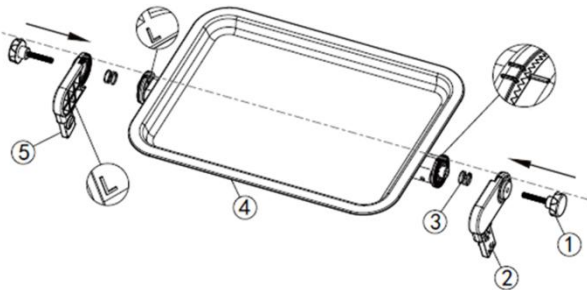

Width Adjustment

Both ends of the tray can be extended left and right to fit the width of the rollator.

Then mount the tray downwards and lock it with button “6” on the latch.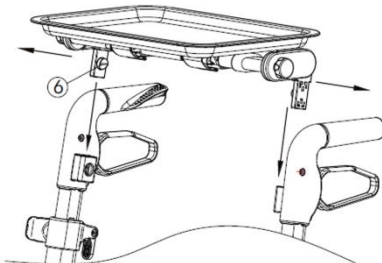


Angle Adjustment and Sliding the Tray

After unlocking screw “1”, the angle of the tray can be freely adjusted and visually checked to ensure the plate is parallel.

Additionally, after unlocking lever “7”, the tray can be moved left and right.

During use, lever “7” must be locked.

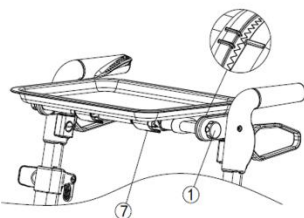IMPORTANT:

In the event of a “serious incident” related to the product that has led, might have led, or could lead to any of the following:

- a) the death of a patient, user, or another person,
- b) a temporary or permanent serious deterioration in their health condition, or
- c) a serious risk to public health,

this “serious incident” must be reported to the manufacturer and the competent authority of the EU Member State where the user or patient resides.

WARNING: If pain, allergic reactions, or other unclear or concerning symptoms occur while using this medical device, consult a healthcare professional.

WARNING: Using the product in any way other than its intended purpose is strictly prohibited.

NOTICES AND WARNINGS

Do not use the rollator to transport people or push it at high speed.

The device is not intended to carry passengers — such use may lead to tipping over and injury.

Do not leave the rollator outdoors for an extended period.

Moisture, intense sunlight, and extreme temperatures can damage metal, plastic, and textile components.

Do not use the rollator on steep, slippery, or highly uneven surfaces without supervision.

This may cause the rollator to tip over and the user to fall.

Do not overload the bag — the maximum load is 5 kg.

Excess weight may cause instability and tipping of the rollator.

Do not exceed the maximum user weight capacity of 136 kg.

Overloading can lead to permanent structural damage.

Always engage the brakes before sitting down or standing up from the seat.

This prevents the rollator from moving unexpectedly.

Do not sit on the rollator while it is in motion.

The seat is intended only for resting when the rollator is stationary and brakes are engaged.

CLEANING**1. General Cleaning**

Clean the frame, handles, seat, and wheels regularly, especially after intensive outdoor use.

Use a soft, damp cloth and mild cleaning agents such as liquid soap or gentle dishwashing detergent.

Avoid harsh chemicals, solvents, alcohol, bleach, or abrasive products that could damage the surface of the frame or plastic components.

After cleaning, dry all parts with a dry cloth, especially metal and moving parts, to prevent corrosion.

2. Cleaning Textile Parts and Bag

If the shopping bag or other textile accessories become dirty, clean them by hand.

Do not dry on a radiator or in a tumble dryer — air drying at room temperature is recommended.

3. Wheel Cleaning and Inspection

Remove sand, mud, stones, hair, and other debris from the wheels that could impair rotation or braking.

If needed, wash the wheels with warm water and mild detergent.

Check the condition of the tires regularly — if they are heavily worn or cracked, they should be replaced.

4. Maintenance of Moving Parts and Brakes

Every 1–2 months (or more frequently with heavy use):

Check the brake performance (ensure wheels are properly locked when brakes are engaged),

Inspect the brake cable tension and adjust it using the adjustment screw at the wheel if necessary,

Ensure all bolts and nuts are properly tightened.

STORAGE

Store the product in a dry place. Do not expose it to direct sunlight.

DISPOSALWhen taken out of service, the product should be treated as standard household waste.

Be sure to follow local waste sorting regulations. Take care of the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawiślanska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawiślanska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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