AT51105/AT51106
INSTRUKCJA OBSŁUGI, CZWÓRNÓG
INSTRUCTION MANUAL, QUAD CANE
GEBRAUCHSANWEISUNG, VIERFUSSGEHHILFE
NÁVOD K POUŽITÍ, REHABILITAČNÍ ČTYŘNOHÝ CHODÍTKO
NÁVOD NA POUŽITIE, REHABILITAČNÝ ŠTVORNOŽKA
MANUEL D'INSTRUCTIONS, CANNE QUADRIPODE
HANDLEIDING, REVALIDATIE VIERPOOT
MANUAL DE INSTRUCCIONES, BASTÓN CUÁDRUPLE DE
REHABILITACIÓN
MANUALE DI ISTRUZIONI, BASTONE QUADRIPODE RIABILITATIVO
INSTRUKTIONSBOK, REHABILITERINGSKVADRUPPEL

MD

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EN

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

USE/INDICATIONS

For people who need support while walking, after lower limb injuries, for balance and weight transfer in traumatic and orthopedic conditions. Also for the elderly, as an aid to maintaining balance while walking.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

-balance problems

-acute inflammation of the skin of the hands, making it impossible to handle the product safely

COMPONENTS:

Quad cane, instruction manual

FEATURES

1. universal (can be used in right and left versions),
2. made of aluminum.
3. foam handle.
4. height adjustment in 2.5 cm increments (maximum height 95 cm, minimum height 70 cm).
5. maximum load of 100 kg.
6. overlay: flexible, black (does not stain parquet floors).
- 7 Lifespan: 5 years from the date of manufacture.
8. weight 0.71 kg.
9. foot spacing: overall width: 15 cm (inside: 10 cm), overall length: 21 cm (inside: 15 cm)
10. diagonal: 22 cm.

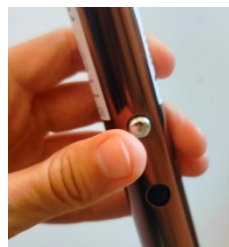
This symbol indicates the maximum weight of the user

USE

Loosen the collet nut to the left (up to two turns).

Use your thumb to press the ball that adjusts the height of the four-foot walking aid.

Move the adjustment tube along the axis to the appropriate height for the user, remembering that the holes in the adjustment tube must be perfectly aligned with the holes in the body.

Move the adjustment tube along the axis to the appropriate height.

Make sure that the adjustment ball has snapped into the correct hole (a distinctive sound may be heard).

Tighten the nut on the collet.

NOTE: Remember to always tighten the collet nut!

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

(a) death of a patient, user or other person or

(b) temporary or permanent deterioration of the health of the patient, user or other person or

(c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

CAUTION: it is forbidden to use the product other than for its intended purpose.

CLEANING

Clean the shaft and handle of the quad cane crutch with a damp cloth and mild detergent.

It is necessary to constantly remove debris from the moving parts of the walker (height adjustment).

Do not use abrasive cleaners!

Do not clean the product with pressure or steam generators!

NOTES: Do not use gasoline-based agents, solvents, acids, chlorine, etc. to wash the product.

STORAGE

Store the product in a dry place, do not expose to direct sunlight.

In order to maintain the performance of the product, it is important to keep all its components clean and observe the method of storage.

Storing the product in humid areas (bathrooms, laundries) or outside leads to a loss of functional and aesthetic parameters of the product.

UTILIZATION

Once the product has been taken out of service, the medical device can be disposed of as normal municipal waste. If in doubt, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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