AT51107
INSTRUKCJA UŻYTKOWNIA, TRÓJNÓG
INSTRUCTION MANUAL, TRIPOD
BEDIENUNGSANLEITUNG, STATIV
NÁVOD K POUŽITÍ, STATIV
NÁVOD NA POUŽITIE, STATÍV
MANUEL D'UTILISATION, TRÉPIED
HANDLEIDING, STATIEF
MANUAL DE INSTRUCCIONES, TRÍPODE
MANUALE D'USO, TREPIEDE
BRUKSANVISNING, STATIV

MD

PL

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EN

Before Use, Carefully Read the Entire Manual

The walking tripod features a single height adjustment mechanism by setting the distance from the handle (grip) to the support point ending in a foot cap (rubber tip). Individual products are adjusted to the user's height by telescopically sliding the tubes together to the desired height and then locking them in place.

The height adjustment is considered appropriate if the user maintains an upright posture while using the product, and the tripod operates vertically, i.e., in a plane parallel to the body axis.

Proper height adjustment and correct operation significantly extend the lifespan of the tripod and ensure the user's safety and satisfaction.

Product Description

Universal (can be used as right-handed or left-handed).

Made of aluminum.

Handle made of plastic.

Incremental height adjustment every 2.5 cm (maximum height 95 cm, minimum height 70 cm).

Maximum load capacity: 100 kg.

Rubber tips: flexible, gray (non-marking on floors and parquet).

Lifespan: 5 years from the date of manufacture.

Weight: 0.75 kg.

Distance between foot caps: 19 cm.

This symbol indicates the maximum user weight.

Intended Use

For individuals needing support while walking, following lower limb injuries, to maintain balance, and to transfer body weight in cases of traumatic or orthopedic conditions.

Also suitable for elderly individuals as a balance aid during walking.

Contraindications

Coordination and balance disorders.

Upper limb injuries or dysfunctions that prevent proper support on the tripod.

Warnings

The tripod can be used indoors and outdoors on even surfaces.

Exercise caution when using the tripod on wet, icy, greasy, or sandy surfaces, as these conditions increase the risk of falling.

Periodically inspect the product visually, checking the height adjustment mechanism and the condition of the foot caps. If any component of the tripod is damaged, discontinue use immediately as it poses a fall risk.

Protect from moisture.

Protect from direct sunlight.

Do not exceed the maximum allowable load (100 kg).

Regularly check the product before and during use to ensure there are no scratches, cracks, or other damages.

Target User Group

Individuals experiencing illnesses, dysfunctions, or injuries for which this product is designed as part of treatment/rehabilitation or compensation. The product may be purchased independently or based on the recommendation of a doctor, therapist, or other specialist.

Whether purchased independently or recommended by a professional, consider the available sizes/features, product variants, indications, contraindications, and information provided by the manufacturer.

User Instructions

Unscrew (loosen) the damping (locking) sleeve nut to the left (maximum two turns).

Press the height adjustment button with your thumb.

Slide the adjustment tube along the axis to the user's desired height, ensuring that the adjustment tube holes align perfectly with the body holes.

Ensure the adjustment ball clicks into the proper hole (audible characteristic sound).

Tighten the damping (locking) sleeve nut.

Note: Do not over-loosen the adjustment tube to the left, as this may cause unwanted locking!

Maintenance

Clean the shaft and handle with a damp cloth using mild detergents.

Regularly remove dirt from moving parts (height adjustment mechanism). Do not use abrasive cleaners.

Do not clean the product with pressure washers or steam cleaners.

Warning: Do not use cleaning agents containing gasoline, solvents, acids, chlorine, etc., to clean the product!

Warning: Do not hang or attach objects to the tripod. Failure to follow this instruction may result in personal injury (e.g., due to a fall) and/or product damage.

To maintain the product's functionality, keep all components clean and properly stored. Storing the product in damp environments (bathrooms, laundry rooms) or outdoors will affect its usability and aesthetic parameters.

Serious Incident Reporting

If a "serious incident" related to the product occurs that directly or indirectly caused, could have caused, or may cause:

- a) death of the patient, user, or another person,
- b) temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) a serious public health threat,

this "serious incident" should be reported to the manufacturer and the competent authority of the user's or patient's country of residence. In Poland, the appropriate authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

Adverse Reactions

If you experience pain, allergic reactions, or other disturbing, unclear symptoms related to the use of this medical device, consult a healthcare professional.

Disposal After Use

After retiring the medical device, it can be disposed of as regular household waste, except for electrical devices, which must be disposed of following proper procedures for electronic and electrical equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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