**AT51117 / AT51117B**

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**Intended use:**

The shower chair (AT51117, AT51117B) is designed for people with reduced mobility, to make it easier for them to use the shower.

The stool legs are height adjustable to ensure maximum comfort for the user. The product has non-slip caps.

After the user has finished using the stool, product's cleanliness shall be ensured by washing it carefully. Standard cleaning and disinfecting agents may be used.

**SPECIFICATION**

colour: white / blue

seat width: 32,5 cm

seat depth: 32,5 cm

seat height: 38 - 51 cm

maximum user weight: 115 kg

weight: 1,5 kg

This symbol means: "Maximum user weight"

**ATTENTION:**

In the event of a product-related

"serious incident" that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- a) death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

This 'serious incident' should be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products..

**ATTENTION:**

In the event of pain, allergic reactions or other distressing symptoms, unclear to the user, related to the use of the medical device, a health professional should be consulted.

**ATTENTION:** Before use, check that all elements have been assembled correctly.

**ATTENTION:** it is forbidden to use the product in a way other than for its intended purpose

**CONTRAINDICATIONS**

physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

**TARGET GROUP OF PATIENTS**

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.


**Installation:**

1. To simplify installation, place the shower stool on a flat surface..
2. Place the legs of the stool on the seat and fasten them with a screw.
3. Place a washer on each hole and secure the stool parts with the remaining screws.
4. You can now insert the legs of the shower stool.

Ensure that the spring pins are in relevant leg holes and legs are at the same height.

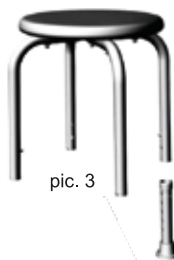Pic. 1

pic. 3

Pic. 2

**Comments:**

1. During use and when assembling and adjusting mechanisms there may be a danger of entrapment and/or compression of parts of the user's/companion's body in openings/gaps between components.
  2. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts firmly.
  3. Before each use of the product, always check that all adjustable parts are securely fastened.
  4. Be careful on wet or slippery surfaces.
  5. Do not insert your fingers into the holes in the seat for draining water. There is a risk of cutting or getting your finger stuck.
  6. Do not use a defective or broken product.
  7. Keep the product out of the reach of children.
  8. Only an authorised service centre may replace missing, damaged or worn parts.
  9. Do not use high-pressure cleaners or aggressive, corrosive cleaning agents for cleaning.
  10. Special care should be taken when using the product.
  11. Check that the product is properly assembled.
  12. Do not lean excessively, especially backwards, as this may result in tipping over.
- The maximum weight of the user is 115 kg

**CLEANING/STORAGE**

The product should be cleaned with delicate detergents, do not use rough cleaning objects. Store at room temperature, in a dry place, away from direct sunlight and moisture.

**DISPOSAL OF THE PRODUCT**

The product must be disposed of in accordance with the applicable disposal conditions.

**NOTE**

The manufacturer is not responsible for damage caused by neglected maintenance, improper servicing or failure to comply with the recommendations contained in this manual.

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**WARRANTY CARD**

Model: .....  
Serial number: .....

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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