## **AT51126**

**KRZESŁO PRYSZNICOWE INSTRUKCJA OBSŁUGI  
SHOWER CHAIR USER MANUAL  
DUSCHSTUHL BENUTZERHANDBUCH  
UŽIVATELSKÁ PŘÍRUČKA K ŽIDLÍČCE DO SPRCHY  
NÁVOD NA POUŽITIE STOLIČKA DO SPRCHY  
MANUEL D'UTILISATION DE LA CHAISE DE DOUCHE  
GEBRUIKSAANWIJZING DOUCHESTOEL  
MANUAL DE USUARIO DE LA SILLA DE DUCHA  
MANUALE D'USO DELLA SEDIA DA DOCCIA  
BRUKSANVISNING FÖR DUSCHSTOL**

**Wstęp**

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**INTRODUCTION**

Thank you for purchasing our product. Please read this user manual carefully before using it. Incorrect use may lead to damage to health, injury or accidents.

Thus, for safe and successful use, please read this manual carefully.

**GENERAL WARNINGS**

DO NOT assemble or use the product before reading and understanding this manual. If you do not understand the warnings, cautions or instructions, contact your healthcare professional, distributor or technician before attempting to assemble the product to avoid injury or damage to the product. Inspect all parts for damage caused in transit. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

**NOTE**

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to, or is likely to lead to any of the following:

- (a) death of the patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above "serious incident" must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

**NOTE**

In case of pain, allergic reactions or other disturbing, unclear to the user symptoms related to the use of the medical device, consult a healthcare professional.

**NOTE**

The manufacturer is not responsible for damage caused by negligent maintenance, inadequate servicing or resulting from failure to follow the recommendations contained in these instructions for use.

**ATTENTION**

It is forbidden to use the product other than in accordance with its intended use!

**ATTENTION**

There may be a risk of overturning if the product is used incorrectly. Please observe the recommendations for sitting on the product. Swinging/rocking on the product is not allowed, as this can lead to the user overturn and cause serious injuries.

**ATTENTION**

During use and handling of the product and when folding and adjusting the mechanisms, there may be a danger that parts of the user's/companion's body are trapped and/or compressed in the openings/gaps between the components. Perform these operations with particular care. Once the adjustment has been completed, stabilise the position by carefully tightening the nuts/bolts.

**PURPOSE**

The toilet chair is intended for disabled persons with musculoskeletal disorders and for elderly people.

**CONTRAINDICATIONS**

Physical or mental limitations that prevent safe handling of the product.

**PRODUCT FEATURES:**

- comfortable seat
- removable armrests for easier use
- lightweight yet robust construction

**ASSEMBLING**

Screw the backrest tubes into the corresponding holes. Then fix the A and B frames with the supplied knobs. Finally, fit the armrests, adjust the appropriate seat height (6 degrees of adjustment in 2.5cm increments) and check that the chair is stable.

**DISASSEMBLING**

Perform all operations in the reverse order of those described in the section , " ASSEMBLING":

**STORAGE:**

Store the product in a dry place, do not expose to direct sunlight.

**MAINTENANCE**

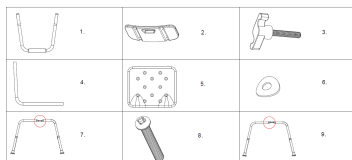
- 1 Ensure that all attachments are always secure.
2. check all fixing screws, tighten as necessary.
3. promptly replace any damaged or deteriorated components.

**CLEANING**

Clean the product with a damp cloth and mild detergent.  
Do not use rough tools or strong detergents for cleaning.

**DESCRIPTION OF COMPONENTS**

1. armrest tubes x2
2. plastic backrest x1
3. backrest fixing knobs x4
4. backrest tubes x2
5. seat x1
6. pads x9
7. frame tube (A) x1
8. frame fixing knobs A and B x5
9. frame tube (B) x1

**TECHNICAL SPECIFICATIONS**

Overall height: 73cm - 85 cm  
Overall width: 51 cm  
Overall length: 49 cm  
Seat width: 40 cm  
Seat depth: 33 cm  
Height of armrests: 19 cm  
Height from floor to seat: 41 cm to 54 cm  
Weight: 5.75 kg  
Maximum user weight: 136 kg  
Frame: aluminium  
Seat and backrest: PE

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER.

**DELIVERY RANGE:** chair (see description of components), user manual

**TARGET GROUP OF PATIENTS**

The healthcare professional indicates the use of the product for adults and children at his own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

**HOW TO DISPOSE OF THE PRODUCT AT END-OF-LIFE**

Once the device has been withdrawn from service, the medical device can be disposed of as normal municipal waste.

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WARRANTY CARD

Model: .....
Serial number: .....
Point of sale stamp

and a legible signature of the Seller: .....

- 1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisławska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic.
Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol.
The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE).
IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisławska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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