**AT51135**  
**INSTRUKCJA UŻYTKOWNIA, CHODZIK**  
**INSTRUCTION MANUAL, ROLLATOR**  
**GEBRAUCHSANWEISUNG, ROLLATOR**  
**NÁVOD K POUŽITÍ, CHODÍTKO**  
**NÁVOD NA POUŽITIE, CHODÍTKO**  
**MANUEL D'INSTRUCTION, DÉAMBULATEUR**  
**GEBRUIKSAANWIJZING, ROLLATOR**  
**MANUAL DE INSTRUCCIONES, ANDADOR**  
**MANUALE DI ISTRUZIONI, DEAMBULATORE**  
**BRUKSANVISNING, RULLATOR**

**MD**

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EN

Thank you for purchasing our product. Please read this user manual carefully. The product must not be used before this manual has been read and understood. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or the seller to avoid damage to the product or personal injury. **WARNING:** Check all parts for damage that may have occurred during transport. If any such damage is noticed, the product **MUST NOT** be used. For more information, contact the seller.

#### **TARGET PATIENT GROUP**

Persons suffering from diseases, dysfunctions, or injuries for the treatment, rehabilitation, or compensation of which this device is intended (see the section on intended use in this user manual).

The product may be purchased by the user independently or based on the recommendation of a physician, therapist, or other specialist. Whether the product is purchased independently or recommended by a physician/therapist/other specialist, the available sizes, required functions, and product variants, as well as indications and contraindications for use and the information provided by the manufacturer, must be taken into account.

#### **INTENDED USE / INDICATIONS**

The product is intended to provide support during mobility for persons with limited walking ability. The rollator is designed for use by one person. The product is suitable for the following purposes:

**Balance training:** The rollator allows patients to practice maintaining balance, which is important in the rehabilitation process after injuries or neurological disorders.

**Posture maintenance:** Patients may use the rollator to train correct body posture, which is especially important for individuals with postural defects or problems maintaining body stability.

**Improvement of motor coordination:** Exercises performed using the rollator may help improve motor coordination and movement control, which is important for people with neurological disorders.

**Support during post-injury rehabilitation:** The rollator may be used in the rehabilitation process following lower-limb injuries, helping patients gradually regain mobility.

**Increasing independence:** For some patients, the rollator may be a tool to increase independence in everyday life by enabling independent movement.

#### **CONTRAINDICATIONS**

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

Persons with serious spinal stability problems or spinal injuries may be unsuitable candidates for using the rollator, as it may increase the risk of further injuries.

In individuals with uncontrolled seizures or epilepsy, the use of the rollator may be dangerous both for the patient and for people around them.

In individuals with very limited muscle strength, use of the rollator may be difficult or ineffective. In such cases, a different rehabilitation approach may be required, such as manual therapy or muscle-strengthening exercises.

In patients with severe balance disorders, use of the rollator may be dangerous, as it may increase the risk of falls and further injuries.

**LIST OF COMPONENTS:** Rollator, user manual

#### **PRODUCT FEATURES**

Can be folded into compact dimensions for easy transport and storage

Equipped with a cane/crutch holder

Nylon bag – protects personal belongings

Suitable for indoor and outdoor use

#### **TECHNICAL SPECIFICATIONS**

Frame: aluminium

Wheel forks: aluminium

Bag: fabric

Seat: foam and woven fabric

Backrest: nylon

Handles: TPR

Overall width: 62 cm

Overall depth: 70 cm

Seat width: 46 cm

Seat depth: 25 cm

Seat height: 53 cm

Seat thickness: 1.5 cm

Distance from the ground to the seat: 53 cm

Distance between handles: 50 cm

Folded dimensions (L x W x H): 70 x 24.5 x 84.5 cm

Adjustable handle height (min–max): 84.5–97 cm

Front wheels: 10" PVC + PP

Rear wheels: 8" PVC + PP

Maximum user weight: 136 kg

Net weight: 11.3 kg

**WARNING!** INDIVIDUAL PRODUCT DIMENSIONS MAY VARY BY  $\pm 2$  CM.

This symbol indicates the maximum user weight.

#### **ASSEMBLY INSTRUCTIONS**

Carefully remove the rollator from the box and remove all protective elements. Place the rollator on all four wheels. Push in and pull outward the tubes supporting the seat until they are secured in the frame holders. You should hear a “click” from the seat locking lever once it is correctly engaged.

Press the button on the handles and adjust them to the desired height. See the bottom of the page: “Determining the correct handle height.”

#### Unfolding

Push in and pull outward the tubes supporting the seat until they are secured in the frame holders. You should hear a click indicating correct locking.

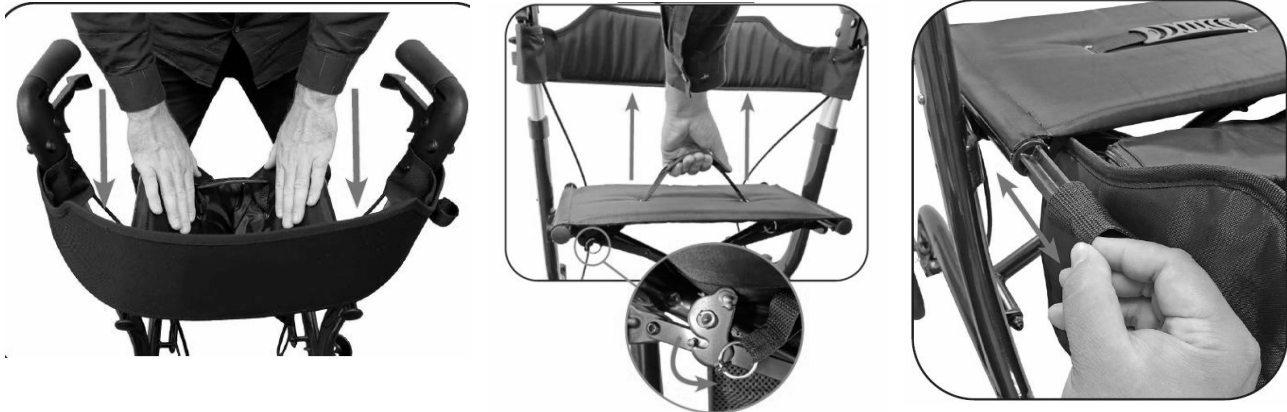
#### Folding

Pull the seat strap upward to release the lever from the mounting under the seat.

Press the seat down and then guide the handles inward.

#### Removing / Attaching the Bag

Loop the nylon strap over the end of the seat support bar. Remove or attach one side at a time.

#### Attaching the Cane Holder

Remove the cane holder from the bag.

Locate the mounting point for the cane holder below the right side of the frame.

Secure the upper part of the cane using the hook-and-loop strap on the right side of the backrest.

## HEIGHT ADJUSTMENT

The handles can be raised or lowered to adjust their height to the user's needs.

To adjust the handle height, press the height adjustment button. Raise or lower the handles to the desired height. You will hear a click when the handle locks into one of the height adjustment positions.

BEFORE USE, MAKE SURE THAT BOTH HANDLES ARE SET TO THE SAME HEIGHT.

DO NOT USE THE ROLLATOR IF THE HANDLES ARE NOT LOCKED IN ONE OF THE HEIGHT ADJUSTMENT POSITIONS.

### How to Properly Adjust the Rollator

Rollators are not universal, therefore it is very important to choose a model that is suitable for your height. Pay attention to two key aspects:

Handle height – for correct posture while walking.

Seat height – should allow safe and comfortable sitting on the rollator, with your back resting against the backrest, feet flat on the ground, and legs bent at a 90° angle.

### How to Sit on the Rollator

Lock the brakes by pushing the brake levers downward until they lock in the lower position. The brakes must remain locked for the entire time you are sitting.

Check the stability of the rollator before sitting down to ensure that it will not move.

Use the handles to stabilize your body while turning around.

Before sitting down, touch the back of your legs against the edge of the seat to make sure you are positioned correctly in relation to the rollator.

Holding the handles, slowly lower your body onto the seat.

DO NOT place your full body weight on the handles while sitting down.

The rollator is intended for stationary sitting only. If a person is seated and pushes off with their feet, the rollator may tip over, causing a fall. The rollator must not be used as a wheelchair and must never be pushed or moved while someone is sitting on it.

### How to Stand Up from the Rollator

Make sure the brakes are locked in the lower position.

Sit comfortably on the edge of the seat.

Position your toes in line with the edge of the seat.

Place both hands on a chair, or one hand on a chair and the other on the rollator.

Never place your full body weight on the rollator, as it may move.

Lean forward until part of your body weight is on your feet.

Stand up using your legs – your hands should support only the weight that your legs cannot lift.

Do not move forward until you are sure of your balance.

Release the brakes and begin walking.

### How to Walk with the Rollator

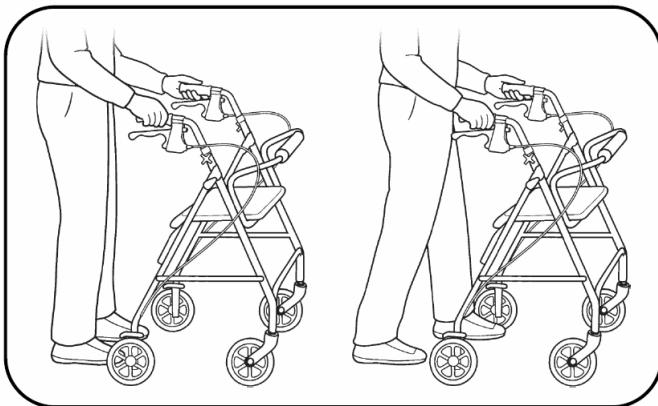
Hold the handles so that your fingers rest around the brake levers in the squeezing position. This allows you to stop the rollator quickly or control your speed by squeezing the brakes if necessary.

Slowly move the rollator forward so that the rear wheels are a few centimeters in front of your body.

Step forward and place your foot in line with the rear wheels.

Maintaining an even pace, move the other foot forward so that it is also in line with the rear wheels.

Establish a walking pace that is comfortable for you and allows each step to be taken in line with the rear wheels.

### Determining the Correct Handle Height

Stand upright behind the rollator with your arms relaxed at your sides and both feet in line with the rear wheels.

Adjust the height of the rollator legs so that the handles are at wrist level.

This adjustment should result in your elbows being bent at an angle of 20–30 degrees when using the rollator.

#### Use and Adjustment of Hand Brakes

To use:

Squeeze the hand brake to stop the rollator or control walking speed.

To lock / unlock:

Push the brake levers downward.

The levers will lock in the lowered position when properly engaged.

Pull the brake levers upward to release.

How to Adjust the Brakes

Note: The braking system on your new rollator has been factory set. To maintain this setting, make sure the lock nuts on the tension adjustment screws are tightened securely against the housing and rear wheels. If you still experience difficulty with the brakes, they can be adjusted as follows:

To tighten the brakes:

Turn the tension adjustment screw counterclockwise – this will move the brake pad closer to the wheel. Continue until the desired tension is achieved.

To loosen the brakes:

Turn the tension adjustment screw clockwise – this will move the brake pad away from the wheel. Continue until the desired tension is achieved. Once the correct tension is reached, retighten the lock nut so that it is firmly pressed against the top of the brake pad.

#### How to Prevent Tipping While Sitting

Distribute your weight evenly on the seat.

DO NOT lean on only one side of the rollator.

Both feet should rest flat on the ground.

DO NOT place your full body weight on the handles while sitting down.

#### **WARNINGS REGARDING THE USE OF THE ROLLATOR**

DO NOT exceed the maximum load of 136 kg.

DO NOT use as a wheelchair or to transport a person.

DO NOT move by pushing yourself while seated on the rollator.

Before use, ensure that all detachable legs, wheels, and other moving parts are in good technical condition. Regularly check the locking buttons in the handles and legs to ensure they are securely engaged.

Regularly check the locking buttons of the backrest to ensure they are locked.

Always consult a physician or therapist before use to determine proper adjustment and correct usage.

Brakes must be locked before using the seat. Do not attempt to push off or walk in any way while seated.

Both feet must rest flat on the floor while sitting.

DO NOT lean against the backrest or extend your body beyond it – it is not designed to support the full weight of the user. The backrest must always be installed when using the rollator.

DO NOT use on stairs, escalators, moving walkways, or uneven surfaces.

DO NOT attempt to reach for objects while seated – this may alter weight distribution and cause the rollator to tip over.

The rollator bag/basket has a maximum load capacity of 4.5 kg. Items must not protrude beyond the edge of the bag.

Ensure that the rollator frame is fully unfolded before use.

DO NOT sit on the rollator when it is positioned on an inclined surface.

DO NOT place your full body weight on the handles when sitting down.

DO NOT remove the warning label.

WARNING: Use of the product in any manner other than its intended purpose is prohibited.

**WARNING:**

In the event of a “serious incident” related to the medical device that has directly or indirectly led, could have led, or may lead to any of the following events:

- a) death of the patient, user, or another person, or
- b) temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) a serious threat to public health,

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

**WARNING:** If pain, allergic reactions, or other disturbing or unclear symptoms related to the use of the medical device occur, consult a healthcare professional.

**WARNING:** Use of the product in any manner other than its intended purpose is prohibited.

#### **CLEANING**

Clean the device regularly and keep it clean. Light dirt and dust can be removed with a damp cloth. Heavier soiling can be removed using a mild, non-corrosive cleaning agent and warm water. Do not use strong abrasives or chlorine-based cleaning agents, as these may cause damage or discoloration. Dry the parts with a soft cloth.

**Wheels:** Clean the wheels with warm water and a mild detergent. Dry with a clean cloth. Lubricate periodically with a multipurpose oil or whenever the wheels begin to squeak.

#### **MAINTENANCE**

To ensure safe and proper use of the product, regularly inspect individual components according to the schedule below:

**Weekly (weekly inspection):**

Check that all screws are securely tightened.

Inspect brake cables for looseness or damage.

Check screws and nuts and tighten if necessary.

**Monthly (monthly inspection):**

Check the condition of the handles to ensure they are stable and undamaged.

Inspect the brake lever to ensure it functions correctly.

Check the condition of the bases (legs) to ensure they are stable and free of play.

**Every four months (quarterly inspection):**

Inspect the wheels to ensure they rotate freely and are not damaged.

Check the bag to ensure it is not damaged and is suitable for use.

Inspect the frame (main frame and supports) for cracks or deformation.

#### **TRANSPORT**

The rollator is easy to fold, making it convenient for transport. If transportation is required, it is recommended to fold the rollator as described above. The rollator should be transported in a manner that ensures stability and prevents sudden or dangerous movement of the device.

#### **REUSE**

The rollator is suitable for reuse. Before being passed on to another user, it must be inspected by a specialized retailer. The following points must be checked:

Visual condition of the frame

Secure fastening of nuts and screws

Condition of plastic components

Condition of the wheels and ball bearings

Effectiveness and condition of the brakes

If any component is damaged or broken, it must be replaced with a new one. Replacement of the wheels is recommended whenever any change is noticed. All loose screws and nuts must be tightened. The product must be thoroughly cleaned and disinfected before reuse.

**WARNING!** Reuse of the rollator is prohibited if the frame is bent or broken.

The instruction manual must accompany the rollator whenever the user is changed.

#### **STORAGE**

Store the product in a dry place and do not expose it to direct sunlight.

#### **DISPOSAL**

After the product is taken out of service, it should be disposed of as regular household waste. Follow local waste segregation regulations.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR MEDICAL Sp. z o.o. ,03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR MEDICAL Sp. z o.o.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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