AT52311
WÓZEK INWALIDZKI INSTRUKCJA UŻYTKOWANIA
WHEELCHAIR USER MANUAL
ROLLSTUHL-BEDIENUNGSANLEITUNG
NÁVOD K POUŽITÍ INVALIDNÍHO VOZÍKU
NÁVOD NA POUŽITIE INVALIDNÝ VOZÍK
MANUEL D'UTILISATION DU FAUTEUIL ROULANT
GEBRUIKSAANWIJZING ROLSTOEL
MANUAL DE USUARIO DE LA SILLA DE RUEDAS
MANUALE D'USO DELLA SEDIA A ROTELLE
BRUKSANVISNING FÖR RULLSTOL

PL

Wstęp

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ENG

Thank you for buying our product! Before using the wheelchair, carefully read the user manual to avoid personal injury or accidents. Your safety is our priority.

GENERAL WARNINGS

- When getting in or out of the wheelchair, make sure that both brakes are engaged.
- Ensure that both brakes are engaged when the wheelchair is not being pushed or used.
- Handle curbs carefully and make turns on hills.
- Steep and uneven surfaces should be avoided as this may cause the wheelchair to tip over.
- Avoid soft surfaces as the wheels may sink in.
- Move the wheelchair smoothly in all conditions.
- Always keep both feet on the footrests and ensure that clothing is properly stowed and does not block the wheels.
- Always drive carefully on inclines.
- Always keep both hands on the push handles to properly control the wheelchair (assistant).
- Always remember to keep your balance.
- Check the wheelchair frame and components regularly and always follow these instructions during and after the warranty period.
- Keep the wheelchair away from sources of fire
- Under no circumstances should the wheelchair be overloaded.
- Avoid hanging heavy objects on the back of the wheelchair to maintain stability, especially when going uphill.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not raise the front wheels while driving, as this may cause a fall and serious injury.
- Do not get into the wheelchair while moving.
- Do not move the wheelchair too fast or make sudden turns.
- Do not stand on the footrests when getting in or out of the wheelchair.
- Do not lift the wheelchair using its moving (removable) parts to prevent injury to the user, caregiver, or damage to the wheelchair.
- Do not hang any objects on the rear connector, as this may cause the backrest to fold down and cause serious injury to the user.

ATTENTION: Exercise caution during wheelchair use, operation, folding, and adjustments to prevent entrapment or pinching of body parts in openings. Perform these steps with care and secure the adjusted position by tightening nuts/bolts carefully.

NOTE: The manufacturer is not liable for damage due to neglected maintenance, improper servicing, or failure to follow manual recommendations.

NOTE: In case of a product-related "serious incident" that has, may have, or may lead to the following:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration of the patient's, user's or other person's health or
- (c) a serious threat to public health

Report such "serious incidents" to the manufacturer and the relevant authority in the user's or patient's residing Member State. In Poland, contact the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

NOTE: If you experience pain, allergic reactions, or other unclear symptoms related to the medical device, consult a healthcare professional.

ATTENTION: it is forbidden to use the product in a way other than for its intended purpose

APPLICATION/INDICATIONS

The wheelchair is designed for individuals with spinal cord injuries, lower limb injuries, neuromuscular diseases, brain and cranial injuries, and geriatric patients. It's also suitable for those who wish to actively participate in daily life but cannot walk independently.

CONTRAINDICATIONS

Avoid sitting position if there are contraindications. Stabilize upper spine and head as needed. Individuals with limitations hindering safe handling, like visual impairment, should refrain. Do not leave patients with mental disorders unattended in the wheelchair.

SECURITY

Entrances, doors

The safest operation of the wheelchair is on a level, flat surface. possible, always use ramps when entering or exiting a building.

Bending over

Do not lean to the side of the wheelchair, as this may cause you to lose your balance and fall.

- Do not make turns on slopes due to the risk of overturning.
- When on a hill, push the wheelchair forward perpendicular to the incline. Descend with the user's back facing the hill to prevent falls.

Curbs

- If possible, always use the help of an attendant (when moving with an attendant, the wheelchair must be equipped with push handles).
- Avoid strong jolts when driving off the curb. Shock can have an adverse effect on the product.
- Exercise caution over curbs and hills, and avoid walking backward in your wheelchair without attendant assistance.

Stairs

- Always use two or more attendants when climbing stairs.
- Do not climb stairs while driving forward.
- Do not use escalators. You must use the elevator.

NOTE: Incorrect use may cause tipping. Follow boarding/dismounting/moving instructions carefully.

DELIVERY RANGE


Wheelchair, two footrests, calf strap, footrest height adjustment wrench, instruction manual

USE

- After unpacking the package, check that it contains all the parts according to the list of items below.
- Make sure that all elements have no visible mechanical damage. In case of doubt, contact the manufacturer.
- Do not use the wheelchair until all its components are properly connected.

UNFOLDING THE TROLLEY

- The wheelchair must be placed with the rear wheels facing you. (picture 1)
- Fold the wheelchair handles outwards and unfold the wheelchair backrest. (Illustration 2)
- Place the wheelchair with the front wheels facing you.
- The seat frames must be pressed down until the wheelchair is fully unfolded. (figure 3)
- Attach footrests by hanging the mounting brackets on the footrest hangers (illustration 4 and 5). Turn inward until the footrest lock clicks into place (illustration 5).
- Make sure the footrests are locked.
- Then unfold the footrest platforms.
- The calf strap should be fastened to the footrest frames. (figure 6)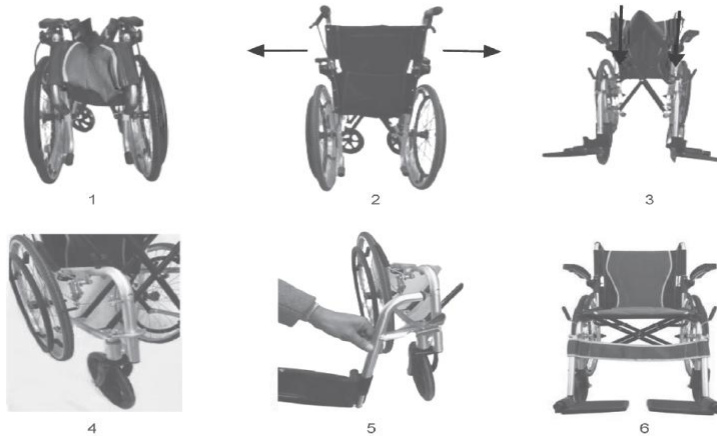

ARMREST


The wheelchair is equipped with a movable armrest.

BRAKE MECHANISM

The wheelchair is equipped with double brakes (user and assistant).

- To activate the brake, pull the brake lever towards the front wheel until the brake is tight against the rear wheel.
- Make sure that the brake lever will not kick back when trying to move the wheelchair.
- To disengage the brake, pull the brake lever towards the rear wheel.
- To change the position of the brake mechanism, loosen two screws from the inside of the wheelchair with a spanner.
- Move the entire mechanism towards the wheel and tighten the screws.
- When the brake is re-applied, the wheel should not turn (test the correctness of the adjustment in the wheelchair with the patient).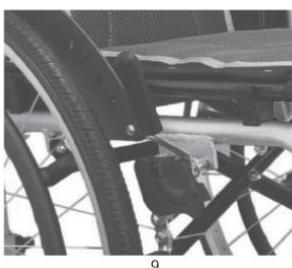


SEAT BELT

The wheelchair is equipped with a seat belt.


FOOTREST

- The footrest is adjustable in length.
- To set the desired length of the footrest, remove the footrests.
- There is a footrest adjustment screw at the bottom of the footrest. (Illustration 11)
- Use the wrench to loosen the footrest adjustment screw. (Illustration 12)
- Adjust the embedded footrest platform in the frame as needed by sliding it in or out.
- Once you choose the right footrest length, secure it by tightening the adjustment screw on the wheelchair footrest.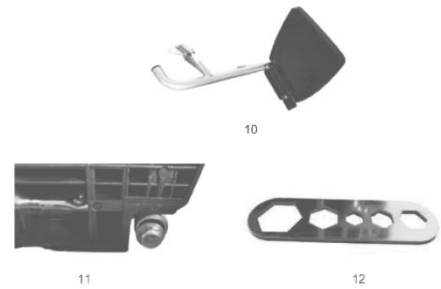

ELEMENTS OF A WHEELCHAIR

1. Push handle
2. Armrest
3. Side panel
4. Brake
5. Calf support straps
6. Step of the footrest
7. Back upholstery
8. Additional brake
9. Seat
10. Cross frame (cross member)
11. Rear wheel
12. Front wheel

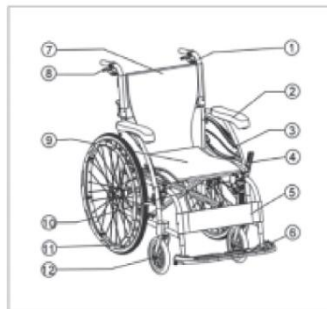PARAMETERS

Frame: aluminum
Maximum load: 115 kg
Total height when folded 71 cm
Total width when folded 38 cm
Total height: 88 cm
Total length: 103 cm
Total width: 61.5 cm
Seat height: 49.5 cm
Seat width: 46.5 cm
Seat depth: 42 cm
Armrest height: 23 cm
Backrest height: 37.5 cm
Solid rear wheel: 21"
Solid front wheel: 7"
Weight: 10.5 kg

THIS MARK MEANS THE MAXIMUM USER WEIGHT!

INSPECTION AND MAINTENANCE

Before driving, inspect brakes for wear or damage; contact the service center for issues. Ensure hand grips are undamaged and secure. Tighten all bolts and nuts, replace worn parts, check alignment, and verify brake operation. Periodically check wheels and tires for cracks or wear; replace through an authorized service. Lubricate wheels with 3-in-1 grease if squeaking occurs.

CLEANING

Prolong product life by cleaning it regularly, especially after use in rainy or snowy weather. Avoid using the wheelchair in rain or snow; if encountered, seek shelter. Clean upholstery and frame with a damp cloth (no strong detergents). Remove debris from moving parts. Regularly clean textiles. To maintain performance, keep all elements clean and store properly. Avoid exposure to extreme weather conditions to prevent color changes.

DO NOT USE:

- solvents, strong disinfectants
- toilet cleaners,
- agents containing chlorine,
- sharp brushes and hard objects,
- aggressive cleaning agents.
- pressure or steam aggregates.

FRAME

Wipe with a damp cloth.

UPHOLSTERY

Clean upholstery with warm soapy water or upholstery cleaner/foam, avoiding strong detergents or solvents. Keep the wheelchair away from fire sources to prevent injury and damage.

TARGET PATIENT GROUP

Individuals with conditions for which this device is intended (refer to instructions) can purchase it independently or based on a doctor's, therapist's, or specialist's recommendation. Regardless of the purchase method, consider available sizes, necessary functions, variants, indications, contraindications, and manufacturer-provided information.

STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

UTILIZATION

The medical device can be disposed of as ordinary municipal waste, please dispose of it in accordance with the applicable general disposal conditions.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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