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GENERAL WARNING Do not use the product before reading and understanding this manual. If you do not understand any warnings, cautions or instructions, please contact your healthcare professional or retailer to avoid damaging the product.

NOTE: Check the product for damage caused during transport. If any such damage is found, DO NOT use the product. More information from the manufacturer

GENERAL INFORMATION Thank you for purchasing our product. The AT53040 and AT53041 brace, through a unique combination of Power Mesh material and innovative Power-Band, provides support, excellent durability, flexibility and comfort as well as proper fit, and the inner layer of breathable silicone keeps the ankle in the right position. It provides strong and effective support for weakened muscles and joints and prevents injuries. The product is designed to provide maximum support while maintaining a comfortable fit. The product is suitable for both everyday use and sports use. It is important to get the correct size to ensure maximum benefit and avoid over-compression. If the orthosis is being used after an injury, see your doctor if there is no improvement within 72 hours. If the injury is severe or if you have any doubts, see your doctor immediately. Adult supervision is required when used by children. Keep out of the reach of small children.

PRODUCT FEATURES AT53041:

- Ultra-thin, breathable material provides support and gentle compression for all-day wear comfort.
- Provides adequate support for the ankle tendon and ligament, increases proprioception during movement.
- Adjustable strap provides comfort and support for the ankle and Achilles tendon.
- Provides support for the main lateral ligament of the ankle and prevents torsion of the ankle joint.
- It helps in quick regeneration.
- Reduces ankle pain.
- Easy to put on and take off.

INDICATIONS AT53041:

- injuries of the ankle joint,
- diseases of the ankle joint,
- rehabilitation after various diseases of the ankle joint,
- a precautionary measure in the event of weakness in the ankle joint,
- post-traumatic changes,
- during the rehabilitation period after surgery.

PRODUCT FEATURES AT53040:

- Ultra-thin, breathable material provides support and gentle compression for all-day wear comfort.
- 3D cut provides good fit and slim leg appearance.
- The silicone Power-Band provides additional compression of the knee joint and increases proprioception during movement.
- Covers the top of the calf and reduces muscle vibration.
- The silicone anti-slip BreathGrip pattern effectively prevents the orthosis from shifting, leaves no imprints on the skin and stimulates proprioception at the contact points with the orthosis.
- Provides support / stability to the knee joint with full freedom of movement for the knee.
- Helps to achieve quick regeneration.
- Reduces pain in the knee.

INDICATIONS AT53040:

- knee injuries,
- diseases of the ligament, patella,
- rehabilitation after various diseases of the knee joint,
- a precautionary measure in the event of knee weakness,
- post-traumatic changes,
- during the rehabilitation period after surgery.

CONTRAINDICATIONS:

- if you get soreness or a rash, stop using it immediately and consult your doctor

PRODUCT CLEANING INSTRUCTIONS:

- The brace can only be washed by hand in lukewarm water (up to 40 ° C) with the addition of a mild detergent (e.g. grey soap).
- Do not iron.
- Do not dry clean.
- Do not use bleach.
- Protect against moisture.
- Do not expose to high temperatures.
- Do not directly expose
- Do not tumble dry.
- Dry after unfolding.

USE:

The brace is put on by pulling it over the leg

SIZES AVAILABLE AT53041: To find the correct size, measure the circumference of the ankle joint:

S: 17-22 cm,
M: 21-25 cm,
L: 24-29 cm.

SIZES AVAILABLE AT53040: To find the correct size, measure the circumference above 14 cm above the knee:

S - 39-45 cm
M - 42-48 cm
L - 45-51 cm
XL-48-54 cm
XXL - 51-57 cm

WARNING: If you are diabetic or have poor circulation, consult your physician before using the product. If you experience pain, rash, or increased pain at any time, discontinue use and consult your doctor.

WARNING: do not use the product if you have used warming lotions or ointments, do not use in the event of skin damage!

REMARKS:

- Please read these instructions carefully before putting on the orthosis.
- Before each use, make sure that the orthosis does not show any abrasions to the seams.
- The use of the product contrary to the principles contained in the instructions may result in serious complications.
- The orthosis should be used according to the instructions of the doctor or physiotherapist.
- The product should not be used in the case of open wounds, abrasions of the epidermis, as well as after previous application of a warming gel or ointment.

CAUTION: Flammable.

ATTENTION: it is forbidden to use the product other than in accordance with its intended purpose!

NOTE: In the event of a product-related "serious incident" which led, directly or indirectly, could or may have led to any of the following events:

- a) death of a patient, user or other person, or
- b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) serious threat to public health

must be reported to the manufacturer and to the competent authority of the Member State where the user or patient is resident.

In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing, or resulting from failure to comply with the recommendations contained in this manual.

NOTE: In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, please consult a healthcare professional.

METHOD OF DISPOSAL OF THE PRODUCT AFTER REMOVING FROM SERVICE

After the device is withdrawn from use, the medical device can be disposed of as normal household waste, with the exception of electrical products - proceed in the manner appropriate for the disposal of electrical and electronic equipment.

STORAGE

The product should be stored in a dry, cool room, away from moisture and direct sunlight.

TARGET GROUP OF PATIENTS

The healthcare professional shall indicate the use of the device to adults and children on his/her own responsibility, taking into account the available variants/sizes/necessary functions/sizes and indications, bearing in mind the information provided by the manufacturer.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisławska 43, hereby grants a 12-month warranty on the roller, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisławska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

Date of issue of the manual: September 16, 2022
v1-1 16/09/2022

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