**AT53112**  
**INSTRUKCJA UŻYTKOWNIA, ORTEZA KCIUKA**  
**USER INSTRUCTIONS, THUMB ORTHOSIS**  
**GEBRAUCHSANWEISUNG, DAUMENORTHESE**  
**NÁVOD K POUŽITÍ, ORTÉZA PALCE**  
**NÁVOD NA POUŽITIE, ORTÉZA PALCA**  
**NOTICE D'UTILISATION, ORTHÈSE DE POUCE**  
**GEBRUIKSAANWIJZING, DUIMORTHESE**  
**INSTRUCCIONES DE USO, ORTESIS DE PULGAR**  
**ISTRUZIONI PER L'USO, ORTESI PER IL POLLICE**  
**BRUKSANVISNING, TUMORTOS**

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EN

Thank you for purchasing our product. Please read the instructions for use carefully. Do not use the product before reading and fully understanding this manual. If you do not understand the warnings, notes, or recommendations, please contact a healthcare professional or the seller to avoid product damage or personal injury. **WARNING:** All parts should be checked for damage that may have occurred during transport. If any damage is found, **DO NOT** use the product. For further information, contact the seller.

**TARGET PATIENT GROUP:** People suffering from diseases, dysfunctions, or injuries for the treatment, rehabilitation, or compensation of which this product is intended (see the section on intended purpose in this user manual). The product may be purchased independently by the user or based on the recommendation of a physician, therapist, or other specialist. Whether purchased independently or recommended by a healthcare professional, the available sizes, required functions, product variants, indications and contraindications for use, as well as the information provided by the manufacturer should be taken into account.

**INTENDED PURPOSE:** The product is intended to stabilize and provide support for the thumb and wrist.

**INDICATIONS:** The product is indicated for use in the following cases: support and stabilization of the thumb and wrist, prevention of overuse injuries, relief of pain in the thumb and wrist area, support of the healing process after soft tissue injuries, overuse conditions and strains of the thumb or wrist, need for light, comfortable support while maintaining full hand functionality.

**CONTRAINDICATIONS:** Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

The product should not be used in the following cases: open wounds, skin damage, or inflammation at the application site, severe injuries requiring rigid immobilization or medical intervention, sensory or circulatory disorders in the hand or wrist area, allergic reactions to the materials used in the product, use inconsistent with the intended purpose of the product. If pain persists, symptoms worsen, or there are any doubts regarding product use, consult a physician or another qualified healthcare professional.

**LIST OF COMPONENTS:** Thumb orthosis, user manual

**PRODUCT FEATURES:** Lightweight design, easy to put on and take off. Provides stable support for the thumb and wrist. Elastomer tapings provide additional, dynamic support. Fits snugly around the sensitive thumb and wrist area, delivering energizing compression and support. Allows free hand movement during daily activities. The product may be worn while washing dishes.

#### TECHNICAL SPECIFICATIONS

Weight: 9 g/Material: 80% nylon, 20% spandex Tapings: 100% polyurethane

Size: S/M and L/XL – for left or right hand

S/M:– thumb circumference: 5–6.2 cm – wrist circumference: 12.5–16.5 cm

L/XL: – thumb circumference: 6–7.5 cm – wrist circumference: 15.5–19.5 cm

**USE:** Slide the orthosis onto the thumb and position it so that the central part of the stabilizer aligns with the axis of the thumb. Wrap the strap around the wrist (from the palm side around). Tighten to a comfortable level of compression and fasten the hook-and-loop closure. Check the fit: the thumb and wrist should be stable, while the fingers can move freely.

**CLEANING:** Wash with the hook-and-loop fastener closed. Hand wash in cool or lukewarm (not hot) water using mild soap. Rinse thoroughly and air dry in a cool place. **DO NOT:** machine wash, tumble dry, dry clean, or use bleach or fabric softeners. **STORAGE:** Store the product in a dry place. Do not expose it to direct sunlight. **DISPOSAL:** After the end of its service life, the product should be disposed of as regular household waste. Follow local waste segregation regulations. Protect the environment.

#### IMPORTANT NOTICE

In the event of a “serious incident” related to the medical device that directly or indirectly led, may have led, or could lead to any of the following:

- a) death of a patient, user, or another person, or
- b) temporary or permanent serious deterioration of a patient’s, user’s, or another person’s health, or
- c) a serious public health threat,

the incident must be reported to the manufacturer and to the competent authority of the EU Member State in which the user or patient resides. **WARNING:** If pain, allergic reactions, or other disturbing or unclear symptoms related to the use of the medical device occur, consult a healthcare professional. **WARNING:** Use of the product in any manner other than for its intended purpose is prohibited.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR MEDICAL Sp. z o.o.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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