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En

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION/INDICATIONS

The hip arthrodesis seat cushion, featuring a replaceable wedge, is designed to relieve pressure on the hip joint for individuals experiencing hip problems or recovering from hip surgeries. This product aids in rehabilitation. The interchangeable wedge can be attached to either the left or right side, allowing the selected side to be lowered and pressure to be alleviated.

The product can be used on various seating surfaces, such as wheelchairs, chairs, armchairs, etc.

INTENDED USE

The intended use of the seat cushion after hip joint arthrodesis is to provide adequate support and comfort during sitting, which helps alleviate pain, reduce pressure on the hip joints, and support proper posture during the recovery period after the procedure.

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling of the product.

-For people with sensory impairments in the lower body, use of the cushion may lead to unnoticed pressure that can cause pressure sores or other skin problems.

- Instability when sitting down and standing up, balance disorders

FEATURES

-interchangeable wedge, allowing adjustment of pressure relief on the selected side of the body

-ease of assembly

-ease of use

-easy to transport

-removable cover available in two functional versions

-zipper for easy handling of the product

SPECIFICATIONS

Seat surface 40 x 40 cm

Foam: polyurethane

Cover available in two versions:

Dimensions of large wedge: 40x40x8cm (W*L*H)

Small wedge dimensions: 19.5x24.4x4.8cm (W*D*H)

Beveling of arthrodesis flap [angle or height front/back] 14°

Density of cushion 36-40 kg/m³

Compression hardness of cushion (CLD) 40% : 5,1-6,9 kPa

Maximum user weight: 130 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

PRODUCT AVAILABLE IN TWO VERSIONS:

AT03024 BASIC VERSION

- Made of a blend of cotton and polyester, which can be washed at 40°C

35% Cotton, 65% Polyester

Weight: 155±10 g/m²

Navy blue color

AT03024D VERSION FOR DISINFECTION AT 95°C

- Made of polyester fabric with PU coating for disinfection at 95°C

63% PES, 37% PU

Weight: 175 [+/-15] g/m²

Turquoise color

COMPONENTS

Cushion with adjustable gusset, Cover compatible with the selected product version, instruction manual

USE

Adjust the position of the wedge according to your needs and relieve the pressure on the joint by sitting comfortably on the cushion. Make sure you don't feel any discomfort or pain.

CAUTION:

In the event of a device-related “serious incident” that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this user manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CLEANING AND CARE

NOTE: DO NOT WASH THE FOAM!

CLEANING THE COVER: unzip the cover with the zipper and remove the foam. The cover can be washed by hand at a maximum temperature of 40°C. Do not iron. Do not dry clean. Do not bleach. Do not tumble dry.

| | |
|---|---------------------------------|
|  | Hand wash at up to 40 degrees C |
|  | Do not bleach |
|  | Do not iron |
|  | Do not tumble dry |
|  | Do not dry clean |

Note: PRODUCT COVERAGE IN 95°C DISINFECTION VERSION, CAN BE DISINFECTED AT TEMPERATURE NOT EXCEEDING 95°C.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

Not suitable for children under three years of age. Store away from sources of fire.

HOW TO DISPOSE OF THE PRODUCT

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste. For its proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

Date of issue of the manual: 22.05.2024
v1-22.05.2024