**AT51051**  
**INSTRUKCJA UŻYTKOWNIA, KRZESŁO TOALETOWE**  
**USER MANUAL, TOILET CHAIR**  
**GEBRAUCHSANWEISUNG, TOILETTENSTUHL**  
**NÁVOD K POUŽITÍ, TOALETNÍ KŘESLO**  
**NÁVOD NA POUŽITIE, TOALETNÉ KRESLO**  
**MANUEL D'UTILISATION, CHAISE TOILETTE**  
**GEBRUIKSAANWIJZING, TOILETSTOEL**  
**MANUAL DE USUARIO, SILLA DE INODORO**  
**MANUALE D'USO, SEDIA WC**  
**BRUKSANVISNING, TOALETSTOL**

MD

## PL

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En  
Thank you for purchasing our product.

Please read the user manual carefully before using the product. Do not use the device before reading and fully understanding this manual. If you do not understand the warnings, notes, and recommendations, please contact a healthcare specialist or the seller to avoid product damage or personal injury.

**WARNING:**

Check all parts for any damage that may have occurred during transport. If such damage is detected, DO NOT use the product. Contact the seller for more information.

**TARGET PATIENT GROUP**

This product is intended for individuals with diseases, dysfunctions, or injuries that require treatment, rehabilitation, or compensation (see the "Product Purpose" section of this manual). The product can be purchased independently by the user or based on a recommendation from a doctor, therapist, or other specialist. Whether purchasing independently or under medical guidance, users should consider available sizes, necessary features, product variations, indications, and contraindications, as well as the manufacturer's provided information.

**APPLICATION/INDICATIONS**

The wheeled commode chair is a medical device designed to assist individuals with limited mobility who have difficulty using a traditional toilet. Its primary purpose is to provide a safe and comfortable solution for toileting and to facilitate daily hygiene, whether in a home setting or a healthcare or long-term care facility.

The chair is intended for daily use by elderly individuals, persons with disabilities, or those recovering from illness or injury, offering necessary support for hygiene-related activities while promoting maximum independence and comfort.

**CONTRAINDICATIONS**

Physical or mental limitations (e.g., visual impairment) that prevent the safe use of the product.

Exceeding maximum load capacity: The chair should not be used by individuals exceeding the maximum weight limit of 180 kg, as this may damage the chair and increase the risk of falls or injuries.

Severe balance disorders: Patients who struggle to maintain balance while seated or exhibit sudden, uncontrolled movements should avoid using the chair without proper supervision and support.

Severe mobility restrictions: Individuals unable to transfer themselves onto the chair or maintain a stable position while using it should only use the chair with the assistance of a caregiver.

Skin conditions or pressure sores: Individuals with existing pressure sores or other skin conditions that may worsen from prolonged sitting should consider additional protective measures, such as special cushions, or consult a healthcare professional before use.

Unstable medical conditions: Patients with unstable health conditions (e.g., fluctuating blood pressure, circulatory disorders, severe neurological diseases) may require specialized equipment and additional supervision while using the commode chair.

**WARNINGS**

Locking the wheels: Before transferring to or using the chair, ensure the wheels are locked to prevent unintended movement and potential falls.

Checking technical condition: Before each use, inspect the chair for any damage, ensuring that all components (e.g., wheels, brakes, frame) are in good working condition. Using a damaged product can lead to accidents.

Avoid using on inclined surfaces: The chair should not be used on sloped surfaces or stairs to prevent tipping over.

Proper use: This product is designed solely as a commode chair. Using it as a wheelchair for prolonged movement may lead to equipment failure and pose a safety risk to the user.

Supervision when needed: If the user requires assistance while using the chair, a caregiver should be present throughout the process to minimize accident risks.

Do not stand on the footrests!

**LIST OF COMPONENTS**

Backrest, Left and right main frames, Footrests, Seat plate, Commode bucket, Rear connecting tube, Front upper and lower connecting parts, Rear lower connecting part, A bag containing screws, locks, an Allen key, and cap nuts, User manual

**FEATURES**

Mobility: Equipped with wheels, the chair is easy to move, allowing convenient transportation of the user to the bathroom or another location where the chair will be used.

Flip-up armrests: Designed for easy transfers from a bed or wheelchair onto the chair, enhancing comfort while sitting and standing.

Stability and safety: The chair is equipped with a wheel-locking system, preventing movement during use and ensuring user safety.

Ergonomics: The chair's design meets the needs of individuals with limited mobility, offering comfortable and secure support during toileting.

Can be used as a shower chair.

Removable footrests and flip-up armrests.

Soft PU seat cushion.

**MATERIAL COMPOSITION**

Plastic with reinforced nylon and fiber materials.

**DIMENSIONS**

Width: 57.5 cm

Height: 97 cm

Length: 86 cm

Seat dimensions: Width 50 cm, Depth 45 cm

Backrest height: 40 cm


Seat height from floor: 52 cm

Wheels: 5 inches

Maximum user weight: 180 kg

This symbol indicates the maximum user weight.  
\* The given dimensions may vary by  $\pm 2$  cm.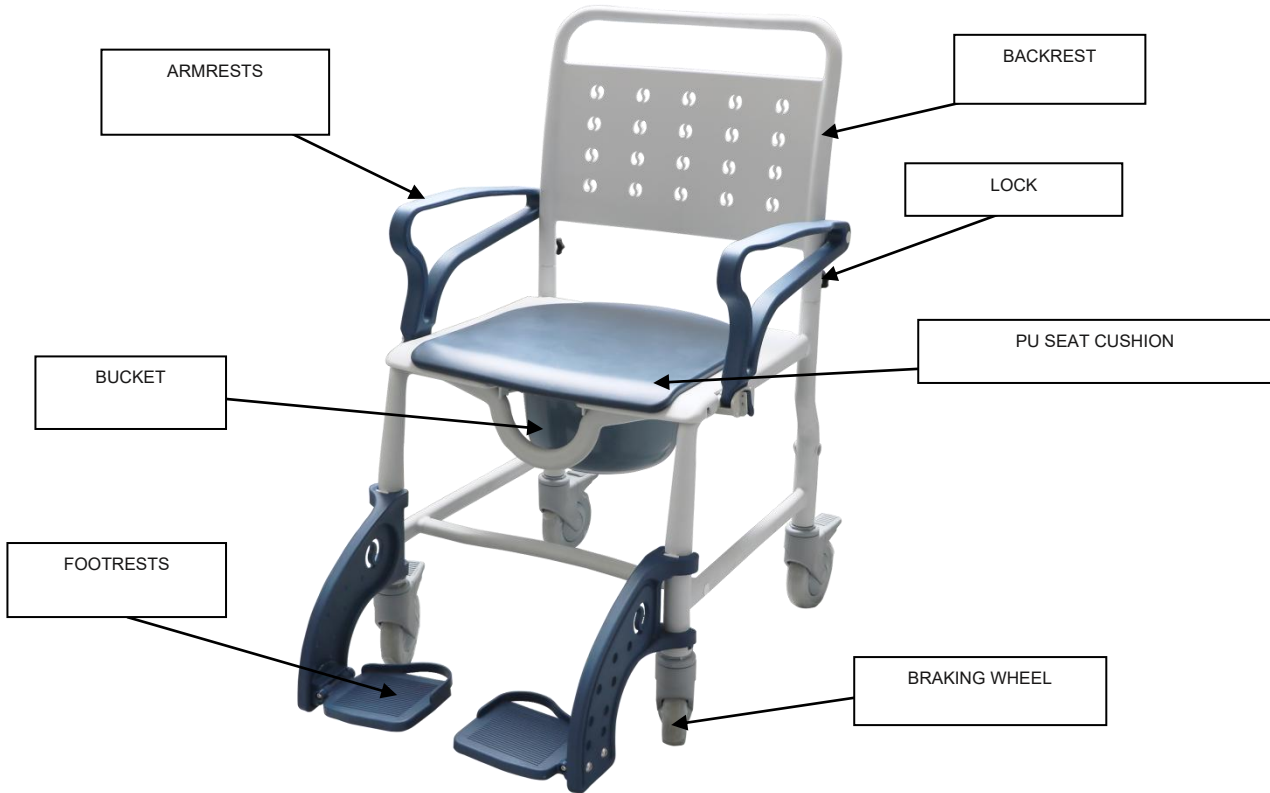

#### ASSEMBLY

Install the main frame:


Place the left part of the main frame and the connecting tubes in their designated positions.

Secure them using screws, then attach the cap nuts to ensure they are properly fastened.


Make sure all connections are tightened securely.

Repeat the steps for the right side:

Install the right part of the frame in the same way as the left side, following the steps above.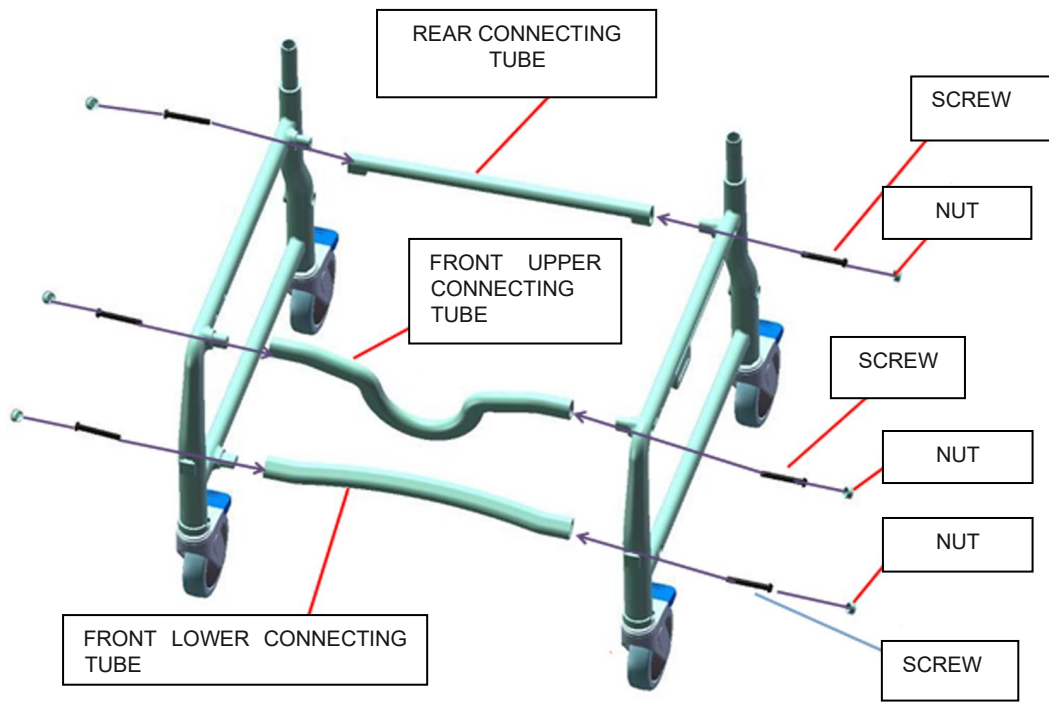

**APPEARANCE AFTER ASSEMBLY**

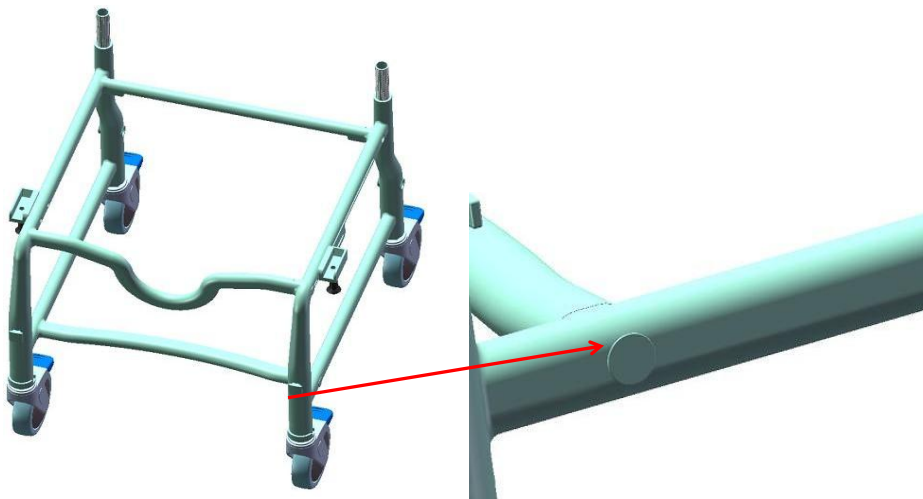ATTACH THE SEAT PLATE TO THE FRAME.

INSERT THE BUCKET INTO THE DESIGNATED SUPPORT TUBES.

Fix the backrest to the upper part of the main frame and secure it with a lock. Lower the armrest and secure it with a locking pin.

Install the footrest onto the tubes with clips.

#### **USAGE**

##### **Before First Use:**

Ensure that all parts of the chair are properly assembled according to the assembly instructions.

Check that all screws, locks, and nuts are securely tightened.

Test the wheel brakes to ensure they function correctly.

##### **Moving the Chair:**

Before transferring the patient to the chair, ensure that the wheels are locked to prevent movement during the transfer.

Make sure the surface on which the chair will be moved is even and stable.

If the patient cannot transfer to the chair independently, assistance from a caregiver is required.

##### **Using the Chair:**

After transferring the patient to the chair, ensure they are in a stable sitting position.

Lower the armrests to provide support and safety.

If needed, the footrest can be folded up or down to enhance the patient's comfort.

The commode chair can be used over a standard toilet or with a bucket, which should be placed in the designated frame holders.

##### **Using the Commode Function:**

Ensure the bucket is properly secured before use.

After using the bucket, gently slide it out of the chair and empty the contents according to hygiene regulations.

Wash and disinfect the bucket before reuse.

##### **Transporting the Patient:**

Before moving the chair, unlock the wheels and slowly push the chair to the designated location.

Avoid sudden movements to ensure the patient's safety and comfort.

When transferring the patient to or from the chair, ensure that the chair is stable and the wheels are locked.

#### **WARNING:**

If a serious incident related to the product occurs, which directly or indirectly has resulted in, could have resulted in, or may result in one of the following events:

- a) Death of the patient, user, or another person
- b) Temporary or permanent deterioration of the health of the patient, user, or another person
- c) Serious public health risk

This serious incident must be reported to the manufacturer and the relevant authority of the country in which the user or patient resides.

In Poland, the appropriate authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

#### **WARNING:**

If pain, allergic reactions, or any other unusual or unclear symptoms occur while using the medical device, consult a healthcare professional.

**WARNING:** The product must not be used in any way other than its intended purpose.

#### **CLEANING AND MAINTENANCE**

To ensure long-term and safe use, it is recommended to regularly inspect all screws and mounting components. Before each use, check that all screws are properly tightened and that there are no signs of loosening.

Regularly check if any parts are missing or damaged. If wear, mechanical damage, or missing components are detected, stop using the product immediately and contact the manufacturer's service to prevent accidents or further damage.

Pay attention to any unusual noises or vibrations, which may indicate loose parts or other technical issues.

When replacing screws or components, only use original parts recommended by the manufacturer.

The product should be cleaned regularly with a soft cloth and water.

**STORAGE**

Store the product in a dry place and avoid direct exposure to sunlight.

**DISPOSAL**

After the product has reached the end of its life cycle, it can be disposed of as regular household waste.

For proper disposal, contact your local waste management authority or the nearest waste collection center to ensure appropriate disposal procedures are followed.

By complying with local regulations, you contribute to the protection of the environment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.  
03-068 Warsaw, ul. Zawisłańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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