AT53044

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General information

The wrist orthosis is made of high-quality fabric, providing excellent strength, flexibility and comfort, as well as a proper fit. The built-in silicone pad provides a gentle pressure massage to the soft tissues, which increases blood flow and accelerates the reduction of swelling. It provides firm and effective support and prevents injury. The product is designed to provide maximum support while maintaining a comfortable fit.

The product is suitable for both everyday use and sports use. It is important to fit the correct size to ensure maximum benefit and to avoid excessive compression. If the orthosis is used after an injury, seek medical advice if the condition does not improve within 72 hours. If the injury is severe or if you have any doubts, seek medical advice immediately. Adult supervision is required for use by children. Keep out of the reach of small children.

NOTES:

In the event of a product-related 'serious incident' that has directly or indirectly led to, could have led to or is likely to lead to any of the following:

- a) the death of a patient, user or other person, or
- b) a temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

WARNING:

If you have diabetes or poor circulation, consult your doctor before using the product. If at any time you experience pain, rash or an increase in pain, discontinue use and consult your doctor.

WARNING: do not use the product if warming liquids or ointments have been applied, do not use in case of skin injuries!

Indications:

- wrist injuries,
- wrist conditions,
- rehabilitation after various wrist disorders,
- a protective measure in case of wrist weakness,
- post-traumatic changes,
- during the rehabilitation period after surgery

Contraindications:

- if soreness or rash occurs, discontinue use immediately and seek medical advice

Features:

- The orthosis ensures function within the optimal anatomical range of mobility.

Product cleaning instructions:

- The brace may only be washed by hand in lukewarm water (up to 40°C) with a mild detergent (e.g. grey soap).
- Do not iron.
- Do not dry-clean.
- Do not use bleach.
- Protect from moisture.
- Do not expose to high temperatures.
- Do not tumble dry.
- Dry unfolded, squeeze gently before drying.

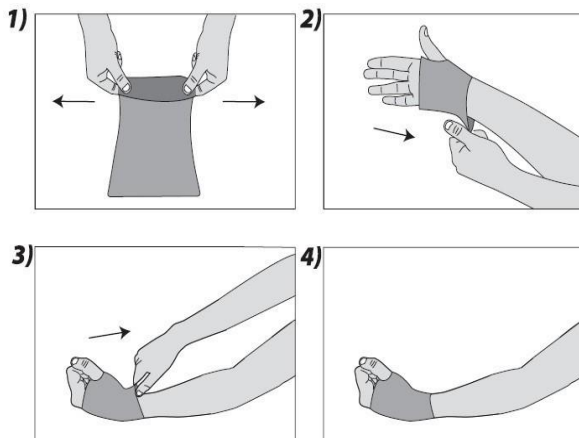
Storage:

Store in a dry area.

How to dispose of the product at end-of-life

Once the device is decommissioned, the medical device can be disposed of as normal municipal waste. The

orthosis is worn by pulling it onto the hand.

To find the right size, measure the circumference of the hand (Figure 5)

Fig. 5

Available sizes: M (17-19 cm), L (19-21 cm), XL (21-23 cm), XXL (23-25 cm)

The brace is available for the right and left hand.

Scope of delivery: Orthosis

Composition: 65% nylon, 15% elastane, 15% latex silk, 5% silicone gasket

Warnings: do not use the product if warming fluids or ointments have been applied, do not use in case of skin damage.

Notes:

- Read the contents of this manual before putting on the orthosis.
- Before each use, make sure there is no abrasion of the seams on the orthosis.
- Use of the product contrary to the instructions may result in serious complications.
- The orthosis should be used as recommended by your doctor or physiotherapist.
- The product should not be used in the case of open wounds, abrasions of the epidermis, or after prior application of a warming gel or ointment.
- Wearing the orthosis at night is only possible if the attending physician so recommends.
- The duration of wearing the orthosis should be consulted with your doctor.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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