**Wanienka do mycia**

**głowy dla**

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**AT51033**

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## En

Head wash tub for disabled AT51033

IT IS MANDATORY TO READ THE CONTENTS OF THIS MANUAL BEFORE USING THE PRODUCT.

STORE AWAY FROM DIRECT SUNLIGHT AND HEAT SOURCE.

REGULARLY CHECK THE EQUIPMENT FOR DAMAGE AND WEAR.

IF ANY DAMAGE OCCURS, STOP USING THE PRODUCT IMMEDIATELY.

**CAUTION:** Check all parts for damage caused in transit. If you notice such damage, DO NOT use the product.

Contact the manufacturer for more information.

**CAUTION:** In the event of a device-related “serious incident” that has directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

**NOTE:** The manufacturer is not responsible for damage caused by negligent maintenance, inadequate servicing, or resulting from failure to follow the recommendations in this manual.

**CAUTION:** It is forbidden to use the product other than in accordance with its intended use!

### **GENERAL WARNING**

DO NOT use the product before reading and understanding this manual. If you do not understand the warnings, cautions, or instructions, contact a healthcare professional, distributor, or technician before attempting to assemble the product to avoid injury or damage to the product.

### Application

The washbasin is intended for washing hair for people with disabilities or limited mobility.

### Installation/use

1 Inflate the washbasin. The valve is non-returnable and does not drain even when left open. To deflate it, press down on the valve and side walls until the air is completely deflated.

2 Place the basin on a flat surface.

3. place the drain hose in the bowl or sink, make sure it is lower than the basin to allow water to flow through.

4. open the drain valve when using the basin to drain water.

5. fill the water bag, the flow of water in the hose is regulated by unscrewing/unscrewing the nozzle.

6. Place the bowl under the user's neck and adjust until it feels comfortable.

### Product features

21 cm deep basin, double reinforced for durability and comfort. Includes a drain valve and hose for quick and easy water removal. In addition, in case of no access to a water source, the product has a water bag with a maximum capacity of 10l.

The product includes: Washbasin with hose, pump, water bag with shower. Material: PVC

### Cleaning

Do not use sharp, rough objects or caustic cleaners (e.g. acids, solvents) for cleaning. All parts are best cleaned with a damp cloth using mild household cleaners. Do not dry on a heater or in direct sunlight.

How to dispose of the product after decommissioning

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste.

### Safety

Use only warm water up to 38 °C to avoid burns! Do not leave the patient unattended in the (filled) basin.

Regularly check the basin for leaks to avoid water damage. Do not use the basin for any purpose other than its intended use.

### Dimensions

Length: 60 cm

Width: 50 cm

Height: 20 cm

Height of head rest: 10 cm Length of drain hose: 70 cm

**NOTE:** All activities while using the product require the assistance of a caregiver.

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Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

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