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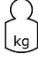
ENG

IMPORTANT:

For the user: Read and keep these instructions before using the device. If you don't understand the warnings, seek help from a healthcare professional or dealer to prevent potential harm. Consult your doctor before using any mobility aid for proper guidance. Dealers should provide these instructions to the user.

SYMBOLS ON THE LABEL OR IN THE INSTRUCTIONS.

	PRODUCER		MAXIMUM PRODUCT LOAD
	PRODUCTION DATE		INFORMATION TO CONSULT THE INSTRUCTIONS FOR USE
	LOT NUMBER		HAVE TO COMPLY WITH SAFETY INSTRUCTIONS SAFETY
	PRODUCT REFERENCE NUMBER		EC CONFORMITY

PURPOSE OF THE PRODUCT

The walking stick is designed for individuals weighing up to 100 kg with limited mobility for walking assistance. It's intended solely for aiding walking indoors and outdoors, not for other purposes.

INSTRUCTIONS FOR USE

Height adjustment

- To change the cane's height, release the safety clip on the locking mechanism (figure B), then use both thumbs to push the mechanism outwards (figure A).
- Adjust the height of the cane by sliding the lower tube into the upper tube or pulling it out - as required.
- After adjusting to the desired height, close the locking mechanism by aligning the two holes of the outer and inner tubes.
- Close the safety clip of the locking mechanism.
- Before fully relying on the adjusted cane, ensure the locking mechanism is securely attached by gradually putting weight on the cane

Adjust the cane's height to fit the user's height properly to avoid limiting its use and durability. For the best height, stand upright, place the cane 15 cm from your foot, and have your arm inclined at roughly 30°. If uncertain, consult a specialist or healthcare professional for guidance.

How to move with the cane?

To be safe, hold the cane as shown in the diagram opposite (2)

Before each use, check that:

- The locking mechanism is correctly fitted and secured (closed safety clip)
- The base/bushing is in perfect condition. A worn base must be replaced.
- If there's any damage, DO NOT use the product. Contact your dealer immediately to prevent potential injury.

2

PRECAUTIONS

Failure to observe the precautions may result in serious injury.

WHEN USING:

- Grip the handle firmly.
- Place the cane on the ground as vertically as possible.
- Do not place the end of the cane too far from your body.
- The maximum load capacity of the cane is 130 kg with a static load; never exceed this weight.
- Use caution when using the cane on carpets, wet, slippery, or snow-covered surfaces. Avoid using the product on icy surfaces.

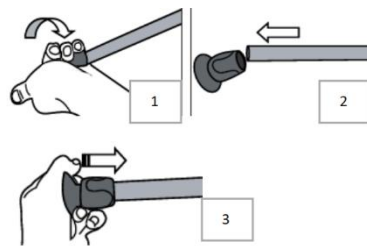
- Beware of holes and grooves in the floor where the cane could get caught.
- Always use the handrail with your free hand on stairs. The use of a third party is strongly recommended to avoid the risk of falling.
- In the event of a fall, try to make the cane fall as far away from you as possible to avoid falling on the cane.
- Never hang bags or other objects on the cane.
- The height adjustment clip is not designed to be removed repeatedly and unnecessarily. Doing so can weaken and break the anti-extension lock.
- Avoid making any mechanical alterations to the cane as it may compromise its safe use. Stick to using only original parts and accessories for the same safety reasons.

CARE AND MAINTENANCE

Lack of care and maintenance of the SOFT STEP cane may cause damage to the product or injury to the user.

- Regularly clean the cane by washing the handles and base with clean water. Wipe all surfaces with a damp, soft cloth, and ensure they are dry before use. Avoid abrasive, greasy, or bleaching products that could make the cane slippery.
- Regularly inspect all cane parts. For safety, have a healthcare professional conduct a thorough check every 6 months. Look for any deformation, corrosion, cracks, or wear on hardware, brackets, and plastic parts.
- Check the condition of the following parts for wear or possible cracks: base (sleeve), handle, locking mechanism. It is important to regularly check the condition of the base of the cane: depending on the surface on which the cane is used, bases will wear faster or slower.
- Replace worn or broken parts promptly for continued safety. Use solely original spare parts available from your dealer or the manufacturer. The only permissible modification is replacing worn parts. If the product remains unused for an extended period, have a healthcare professional conduct a recheck before resuming use.

REPLACEMENT OF WORN PARTS

Replacing the base/base

- To remove the old base, firmly hold the cane in one hand and unscrew the base using the other hand (A and B).
- Push the new base onto the cane with your hand (C).
- Set the cane on the floor and apply gradual pressure until it slides into the base.
- Make sure the base is correctly inserted before using the cane.

Replacing the locking mechanism for adjusting the cane

- The procedure is described in detail in Chapter 3 of the User's Manual, 'Adjusting the height'. "Height The cane is a personal item and has been specially adapted for use by one person. Do not use a cane belonging to another person as there may be a risk of injury..

STORAGE

Failure to comply with the storage conditions may damage the cane and thus pose a risk of injury.

- Do not store the product in a place directly exposed to extreme heat (e.g. by a window or near a radiator) or low temperatures.
- Ensure handles are stable by checking for rotation if exposed to extreme temperatures, high humidity, or moisture. Ensure all parts are tightly secured before use and dry any wet areas to prevent damage or injury.
- If exposed to extreme temperatures - above 41°C - the cane will heat up and first degree burns may result. Let the cane cool down before use.
- Store away from fire and flame sources.
- Observe the storage conditions for the cane outlined below:
 - Store in a dry room at a temperature between 5°C and 41°C.
 - The relative humidity of the air should be between 30% and 70%.
- Protect the cane from dust, corrosion (abrasive elements, sand, seawater, salt air) or oily environments by storing it in its cover.

UTILISATION

Our canes are made of non-toxic, non-biodegradable aluminum and plastic. Please observe local waste disposal regulations and consider returning the product to a recycling facility after use.

TECHNICAL SPECIFICATIONS

Check from time to time that the label on the cane is legible. If necessary, ask your dealer:

- 1 Handle
2. handle height
- Locking mechanism
4. base/sleeve

SOFT STEP MODEL

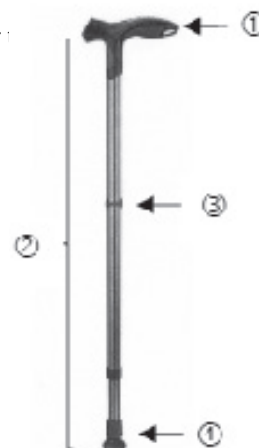
- Handle height (min.- max.): 780-1070 mm
- Support height adjustment: 14 positions in 25 mm increments
- Weight of cane: 360 g
- Maximum user weight Max. 130 kg

SPARE PARTS:

- Locking mechanism
- Base/bushing

ACCESSORIES:

- tripod
- Wrist strap

DURABILITY

The durability of the canes depends on usage, storage conditions, and regular care. Follow the manual guidelines for maintaining your canes in excellent condition.

WARRANTY

The product has a 12-month warranty from the date of purchase, covering latent manufacturing defects. Proof of purchase is required. Normal wear and tear, damage due to improper handling, and failure to follow the user manual's instructions are not covered.

CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

Report the mentioned 'serious incident' to the manufacturer and the relevant authority in the user's residing Member State. In Poland, the designated authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, a healthcare professional should be consulted.

TARGET PATIENT GROUP

Those with medical conditions for which this device is designed (refer to the intended use section in the instructions) can acquire the device personally or upon recommendation from a healthcare professional. When obtaining the device, consider its sizes, functions, variants, indications, contraindications, and the manufacturer's provided information, whether recommended by a healthcare professional or bought independently.

NOTE: it is forbidden to use the product other than for its intended purpose.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślanska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 AN TAR Sp. J.
03-068 Warsaw, ul. Zawiślanska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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